



# How to use the Medline new generation Skin Stapler

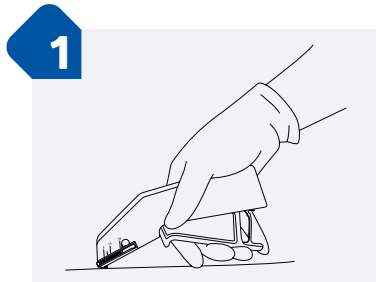


Medline's new generation Skin Staplers are designed to provide an intuitive and reliable wound closure solution, ensuring trigger stability for controlled stapling. This solution is:


- Available in two designs (regular and wide)
- Lightweight and ergonomic, with a slim handle to reduce hand fatigue
- Equipped with 35 stainless steel medical staples
- Designed with an anti-jamming system for smooth application

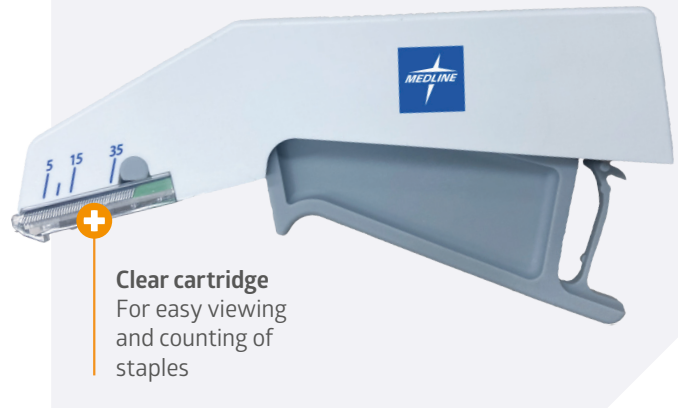


**1** Evert and approximate the skin edges as desired. This can be accomplished with forceps, tissue approximating device, or by applying tension to either end of the incision.

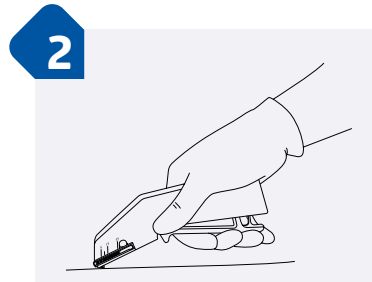

**2** Position the instrument gently over the skin edges. Instrument should be held so that the lip in Figure 1 is aligned parallel to the skin. As the handle is squeezed, an anvil automatically bends the staple to the proper configuration.

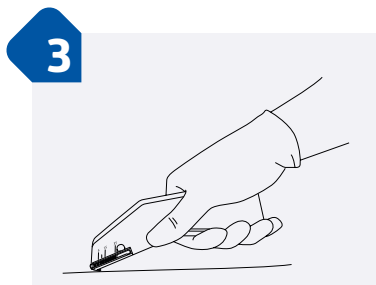

**3** Squeeze trigger completely, there will be an audible "click" heard when the trigger is fully activated. Then completely release the trigger before lifting from the skin.

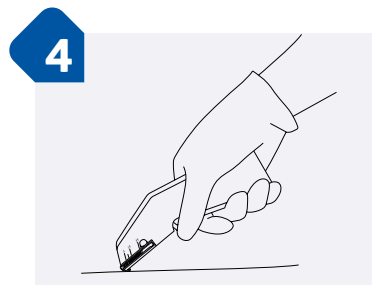

**4** Move the device to the desired location of the next staple and repeat.

Item number	Description		Packaging
STAPLER-35W	Surgical Skin Stapler with progressive trigger, 35 wide staples (Staple dimension after closure: 6.5x3.8 mm)		6/box, 60/case
STAPLER-35R	Surgical Skin Stapler with progressive trigger, 35 regular staples (Staple dimension after closure: 5.8x4.1 mm)		5/box, 100/case

**For more information** about this product or to receive a product demonstration, please contact your Medline account manager or visit our website: [www.medline.eu/uk](http://www.medline.eu/uk)