

How to Use the 24-Hour Kit

Instructions for use: Q800



There is an extra suction swab in every toothbrush tray. Use it to remove secretions before brushing the teeth and to provide additional cleaning when needed.



1 Pull the hanging tab on the top of the box and hang up the box. Open the box as indicated and dispense a tray.



2 With gloved hands, peel back the film. Ensure the foam on the suction toothbrush and/or suction swab and DenTips® is intact. Connect the suction toothbrush/swab to standard suction lines. Ensure the connection is secure.



3 Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands. Saturate the toothbrush/swab with water or a desired solution.



4 Gently clean the teeth and gums for approximately 1 to 2 minutes. Provide suction by using the thumb port.



5 Open the Medline mouth moisturiser and squeeze it onto DenTips. Liberally coat the inside of the mouth (teeth, lips, tongue and palate).



6 Ensure the foam is intact before discarding the suction toothbrush/swab and DenTips. Inspect the oral cavity and remove any particles, if present.



7 Repeat the sequence every 8 hours until all trays are dispensed.

To learn more about the 24-Hour Kit, contact your Medline account manager, or visit us at: uk.medline.eu