



# NATIONALLY CONTRACTED PRODUCT DISPOSABLE TOURNIQUETS


## Indications

Medline Disposable Tourniquets are used on patients for venous blood sampling and intravenous cannulation.

## Why the Medline Disposable Tourniquet?

- Nationally contracted product offering outstanding **value, quality and ease of use**
- Comfortable **latex-free** material that stays in place but can be released with one hand
- **Reduces the risk of cross-contamination** by promoting a single-use policy

## Instruction for Use

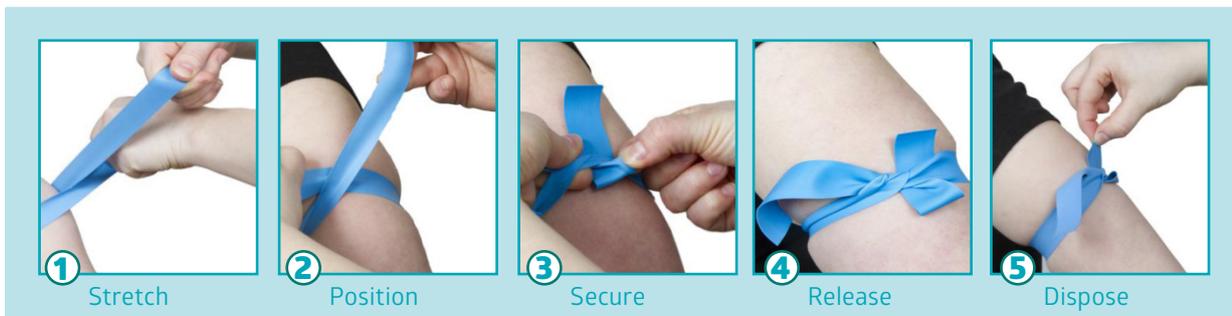

1. Place the tourniquet around the arm, cross the tourniquet and stretch.
2. Continue stretching the tourniquet, hold one end of it down, take the other end and then loop it back under where the tourniquet straps cross.
3. When forming the bow, one end should be longer than the other. The longer end should cross above the shorter. This allows for the tourniquet to be tucked under when making the bow. Do NOT tie a knot as removal will be challenging and may cause discomfort to the patient. Tuck one end under the other so the tourniquet remains securely in place.
4. Remove the tourniquet by pulling the end that is tucked under as indicated in photo 5.
5. Dispose of the tourniquet in the clinical waste bin after use.



Item No.	NPC	Description	Size	Packaging
DYNDE75020A	FWJ100	Disposable rolled tourniquets	2.5 x 45.7 cm	100/cs