



VAPREVENT™

ORAL CARE AND INFECTION PREVENTION

How to Use:

SUCTION SWAB MDS096525EU

The suction function aids in the removal of secretions, and the angled head assists you in reaching the back of the mouth for a more thorough cleaning.

- 1** With gloved hands, check the mouth of the patient and provide global suction of the oral cavity if necessary.
 - » With a suction cannula, for example


- 2** Remove the packaging and check the condition of the swab.
 - » Foam, handle, connector ...

- 3** Connect the suction swab to the suction tubing system and use the thumb port for suction control.
 - » Plug the hole with your thumb for suctioning.
 - » Release the hole to stop the suction.

- 4** Saturate the swab with an appropriate solution.
 - » Water, mouthwash etc.


- 5** Perform the care for 1 to 2 minutes.
 - » The foam part stimulates the oral tissue.
 - » Dip the swab in the solution again if necessary.

- 6** When finished, check the oral cavity of the patient and remove any possible debris before discarding the swab.

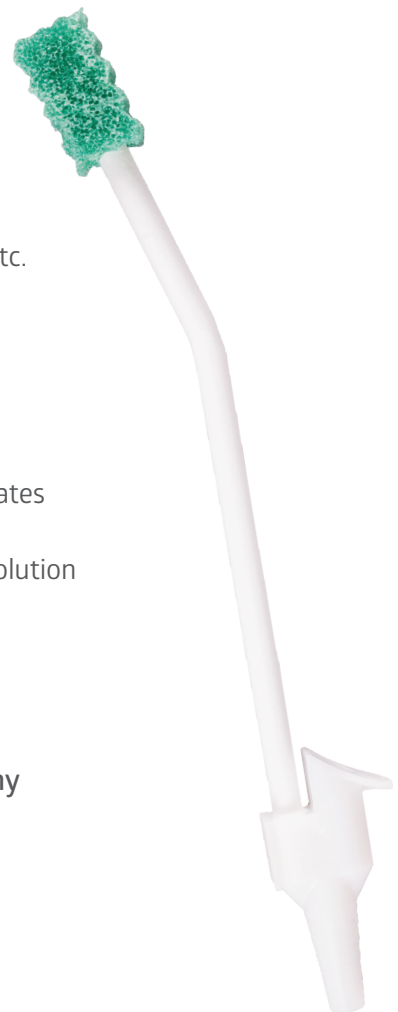

This product is a class I medical device intended to be used by healthcare professionals. Before use, consult instructions and precautions on the corresponding labelling.

  Medline Industries Inc.
Three Lakes Drive
Northfield, IL 60093, USA

 Medline International France SAS
5, rue Charles Lindbergh
44110 Châteaubriant, France

To learn more about oral care, contact your account manager or visit us at: www.medline.eu/uk



VAPREVENT™

ORAL CARE AND INFECTION PREVENTION

How to Use:

SUCTION TOOTHBRUSH MDS096575EU

The super-soft toothbrush with a sponge is used for mechanical debridement and stimulation of the teeth and gums. The suction function helps remove secretions.

1 With gloved hands, check the mouth of the patient and provide global suction of the oral cavity if necessary.

- » With a suction cannula, for example

2 Remove the packaging and check the condition of the toothbrush.

- » Foam, handle, connector ...

3 Connect the suction toothbrush to the suction tubing system and use the thumb port for suction control.

- » Plug the hole with your thumb for suctioning.
- » Release the hole to stop the suction.


4 Saturate the toothbrush with an appropriate solution.

- » Water, mouthwash etc.

5 Perform the care for 1 to 2 minutes.


- » Use the bristles for brushing.
- » The foam part stimulates the oral tissue.
- » Dip the toothbrush in the solution again if necessary.

6 When finished, check the oral cavity of the patient and remove any possible debris before discarding the toothbrush.

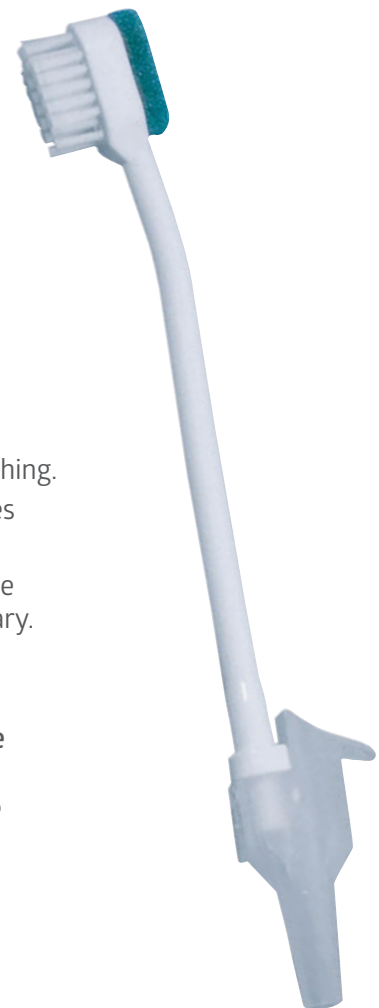

This product is a class I medical device intended to be used by healthcare professionals. Before use, consult instructions and precautions on the corresponding labelling.

CE  Medline Industries Inc.
Three Lakes Drive
Northfield, IL 60093, USA

EC REP  Medline International France SAS
5, rue Charles Lindbergh
44110 Châteaubriant, France

To learn more about oral care, contact your account manager or visit us at: www.medline.eu/uk