



**VAPrevent<sup>®</sup>**  
Oral care  
and infection  
prevention

## How to Use: Suction Swab MDS096525EU

The suction function aids in the removal of secretions, and the angled head assists you in reaching the back of the mouth for a more thorough cleaning.

- 1** With gloved hands, check the mouth of the patient, and provide global suction of the oral cavity, if necessary.
  - With a suction cannula, for example
- 2** Remove the packaging and check the condition of the swab.
  - Foam, handle, connector ...
- 3** Connect the suction swab to the suction tubing system and use the thumb port for suction control.
  - Plug the hole with your thumb for suctioning.
  - Release the hole to stop the suction.
- 4** Saturate the swab with an appropriate solution.
  - Water, mouthwash etc.
- 5** Perform oral care for 1 to 2 minutes.
  - The foam stimulates the oral tissue.
  - Dip the swab in the solution again, if necessary.
- 6** When finished, check the oral cavity of the patient, and remove any possible debris, before discarding the swab.



To learn more about oral care, contact your account manager or visit us at: [www.medline.eu/uk](http://www.medline.eu/uk)



**VAPrevent<sup>®</sup>**  
Oral care  
and infection  
prevention

## How to Use: Suction Toothbrush MDS096575EU

The super-soft toothbrush with a sponge is used for mechanical debridement and stimulation of the teeth and gums. The suction function helps remove secretions.

- 1** With gloved hands, check the mouth of the patient, and provide global suction of the oral cavity, if necessary.
  - With a suction cannula, for example
- 2** Remove the packaging and check the condition of the toothbrush.
  - Foam, handle, connector ...
- 3** Connect the suction toothbrush to the suction tubing system, and use the thumb port for suction control.
  - Plug the hole with your thumb for suctioning.
  - Release the hole to stop the suction.
- 4** Saturate the toothbrush with an appropriate solution.
  - Water, mouthwash etc.
- 5** Perform oral care for 1 to 2 minutes.
  - Use the bristles for brushing.
  - The foam stimulates the oral tissue.
  - Dip the toothbrush in the solution again, if necessary.
- 6** When finished, check the oral cavity of the patient, and remove any possible debris before discarding the toothbrush.



To learn more about oral care, contact your account manager or visit us at: [www.medline.eu/uk](http://www.medline.eu/uk)