

# VAPrevent®

Instructions for use: MDS096013MEU



1 With gloved hands, check the patient's oral cavity and provide global suction if necessary. Then, peel back the packaging film, remove DenTips® and gently ensure the DenTips foam is intact.




2 Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands. Open the Medline mouth rinse and pour it into the cup of the packaging.



3 Saturate the DenTips foam with mouthwash. Gently clean teeth and gums for approximately 1 to 2 minutes.



4 After use, ensure the foam is intact before discarding DenTips. Inspect the oral cavity and remove any particles, if present.

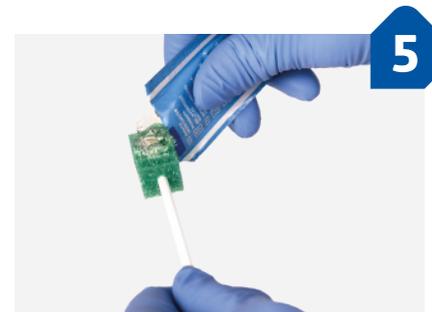

5 Open the Medline mouth moisturiser and squeeze it onto DenTips. Gently coat the inside of the mouth (teeth, lips, tongue, palate) with it.



6 After use, ensure the foam is intact before discarding DenTips. Inspect the oral cavity and remove any particles, if present.

To learn more about oral care, contact your Medline account manager, or visit us at: [www.medline.eu/uk](http://www.medline.eu/uk)