

# VAPrevent<sup>®</sup>

Instructions for use: MDS096013MEU



1 With gloved hands, check the patient's oral cavity and provide global suction if necessary. Then, peel open the packaging film, remove the DenTips<sup>®</sup> and gently ensure the DenTips foam is intact.



2 Use a bite block when performing oral care on patients with altered levels of consciousness, or those who cannot comprehend commands. Open the Medline mouth rinse and pour it into the cup of the packaging.



3 Saturate the first DenTips foam with mouthwash. Gently clean the patient's teeth and gums for approximately 1 to 2 minutes.



4 After use, ensure the foam is intact before discarding the DenTips. Inspect the oral cavity and remove any particles, if present.



5 Open the Medline mouth moisturiser and squeeze it onto the second DenTips. Use it to gently coat the inside of the mouth (inner lips, teeth, tongue and palate).



6 After use, ensure the foam is intact before discarding the DenTips. Inspect the oral cavity and remove any particles, if present.

To learn more about oral care, contact your Medline account manager, or visit us at: [www.medline.eu/uk](http://www.medline.eu/uk)