


VAPrevent®

Instructions for use: MDS096513MEU

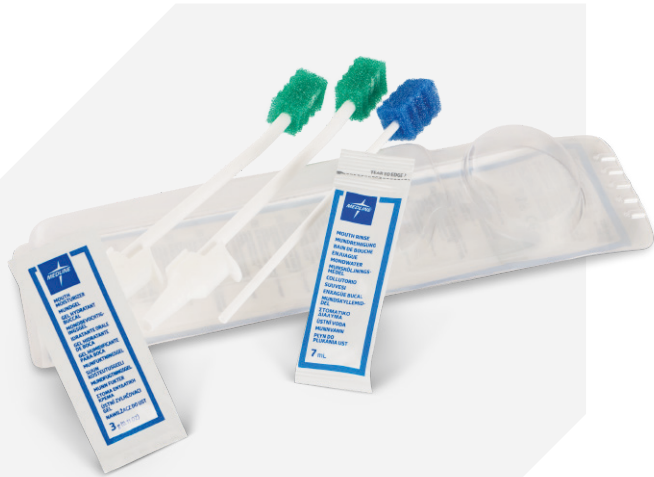

With gloved hands, check the patient's mouth and provide global suction of the oral cavity if necessary. Then, peel open the packaging film and ensure the foam on the suction swabs and DenTips® is intact.



Use a bite block when performing oral care on patients with altered levels of consciousness, or those who cannot comprehend commands. Open the Medline mouth rinse and pour it into the cup of the packaging.



Connect the suction swab to a standard suction line. Check for a secure connection.



Gently clean teeth and gums for approximately 1 to 2 minutes. Provide suction by using the thumb port.



Open the Medline mouth moisturiser and squeeze it onto the DenTips. Use it to liberally coat the inside of the mouth (inner lips, teeth, tongue and palate).



Ensure the foam is intact before discarding the suction swabs and DenTips. Inspect the oral cavity and remove any particles, if present.

To learn more about oral care, contact your Medline account manager, or visit us at: www.medline.eu/uk