

# VAPrevent®

Instructions for use: MDS096572MEU



With gloved hands, check the patient's mouth and provide global suction of the oral cavity if necessary. Then, peel back the packaging film and ensure the foam on the suction toothbrush and DenTips® is intact.



Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands. Open the Medline mouth rinse and pour it into the basin.



Connect the suction toothbrush to a standard suction line. Check for a secure connection.



Saturate the toothbrush with mouth rinse. Gently brush teeth and gums for approximately 1 to 2 minutes. Provide suction by using the thumb port.



Open the mouth moisturiser and squeeze it onto DenTips. Liberally coat the inside of the mouth (teeth, lips, tongue, palate) with it.



Ensure the foam is intact before discarding the suction toothbrush and DenTips. Inspect the oral cavity and remove any debris, if present.

**To learn more about oral care,** contact your Medline account manager, or visit us at: [www.medline.eu/uk](http://www.medline.eu/uk)