

VAPrevent®

Instructions for use: MDS096013MEU



With gloved hands, check the patient's oral cavity and provide global suction if necessary. Then, peel open the packaging film, remove the DenTips® and gently ensure the DenTips foam is intact.



Use a bite block when performing oral care on patients with altered levels of consciousness, or those who cannot comprehend commands. Open the Medline mouth rinse and pour it into the cup of the packaging.



Saturate the first DenTips foam with mouthwash. Gently clean the patient's teeth and gums for approximately 1 to 2 minutes.



After use, ensure the foam is intact before discarding the DenTips. Inspect the oral cavity and remove any particles, if present.



Open the Medline mouth moisturiser and squeeze it onto the second DenTips. Use it to gently coat the inside of the mouth (inner lips, teeth, tongue and palate).




After use, ensure the foam is intact before discarding the DenTips. Inspect the oral cavity and remove any particles, if present.

To learn more about oral care, contact your Medline account manager, or visit us at: www.medline.eu/uk

VAPrevent®

Instructions for use: MDS096513MEU

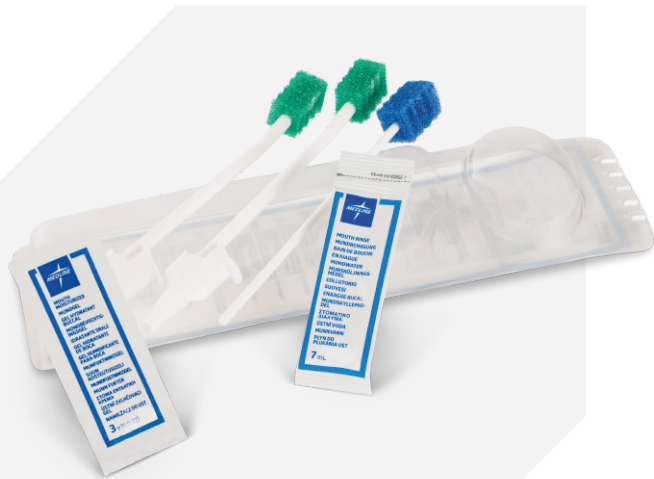

1 With gloved hands, check the patient's mouth and provide global suction of the oral cavity if necessary. Then, peel open the packaging film and ensure the foam on the suction swabs and DenTips® is intact.



2 Use a bite block when performing oral care on patients with altered levels of consciousness, or those who cannot comprehend commands. Open the Medline mouth rinse and pour it into the cup of the packaging.



3 Connect the suction swab to a standard suction line. Check for a secure connection.



4 Gently clean teeth and gums for approximately 1 to 2 minutes. Provide suction by using the thumb port.



5 Open the Medline mouth moisturiser and squeeze it onto the DenTips. Use it to liberally coat the inside of the mouth (inner lips, teeth, tongue and palate).



6 Ensure the foam is intact before discarding the suction swabs and DenTips. Inspect the oral cavity and remove any particles, if present.

To learn more about oral care, contact your Medline account manager, or visit us at: www.medline.eu/uk

VAPrevent®

Instructions for use: MDS096572MEU



With gloved hands, check the patient's mouth and provide global suction of the oral cavity if necessary. Then, peel open the packaging film and ensure the foam on the suction toothbrush and DenTips® is intact.



Use a bite block when performing oral care on patients with altered levels of consciousness, or those who cannot comprehend commands. Open the Medline mouth rinse and pour it into the cup of the packaging.



Connect the suction toothbrush to a standard suction line. Check for a secure connection.



Saturate the toothbrush with mouth rinse. Gently brush the patient's teeth and gums for approximately 1 to 2 minutes. Provide suction by using the thumb port.



Open the mouth moisturiser and squeeze it onto the DenTips. Use it to liberally coat the inside of the mouth (inner lips, teeth, tongue and palate).



Ensure the foam is intact before discarding the suction toothbrush and DenTips. Inspect the oral cavity and remove any particles, if present.

To learn more about oral care, contact your Medline account manager, or visit us at: www.medline.eu/uk