

Five steps to successful humidification

1



Remove the AquaPak and humidification adaptor from their packaging.

2



Screw the humidification adaptor onto the AquaPak bottle.

3



Break off the small plastic nipple on the bottle by lifting it upwards.

WARNING: Do not twist the nipple. Twisting the nipple shrinks the inner diameter of the AquaPak opening and reduces the pressure release, resulting in over-pressurisation of the bottle.

4



Screw the combined AquaPak bottle and humidification adaptor onto the flowmeter.

5



Attach the O2 tubing, including the interface (mask/nasal cannula), to the port.