


Dermadine Plus and Dermadrape Incise Films

Increased sterility for improved safety



**Sterile barrier
system for
surgical
incisions**

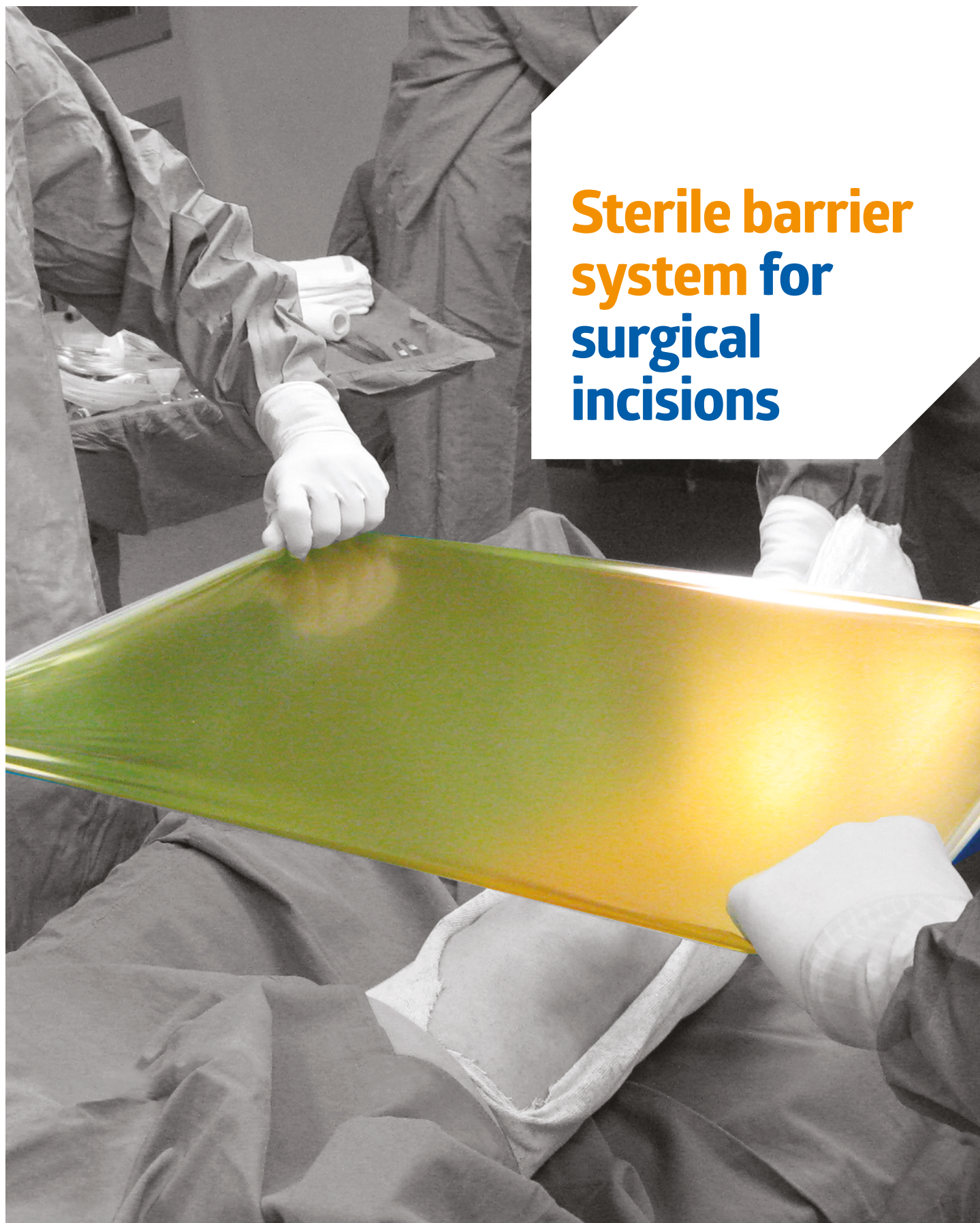

Dermadine Plus Incise Film

Cleaning, in preparation for surgery, and skin disinfection, are two necessary steps for the initial reduction of surface bacteria. However, skin microorganisms are reduced but not eliminated, and can survive in microscopic folds of the skin.

When complete and long-lasting asepsis is required, the Dermadine Plus incise film guaranteed to maintain a sterile field, through its broad-spectrum antimicrobial activity on many microorganisms present on the skin.


The adhesive contains slow-release iodine, which ensures maximum sterility is achieved and sustained over time. It is also suitable for all types of surgical procedures.

Dermadine should, therefore, be used as a final step in preparing the surgical field.




Long lasting

The constant presence of iodine for a considerable time, even beyond 24 hours, makes the product suitable for long surgical procedures.

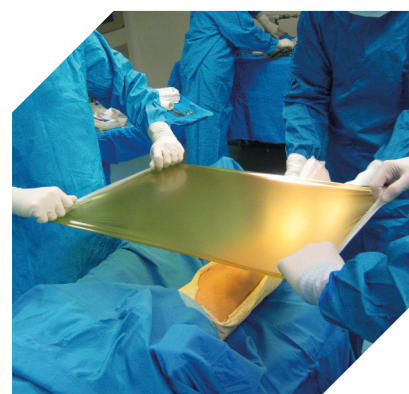

Measurement of antibacterial activity (per ISO 22196:2011)

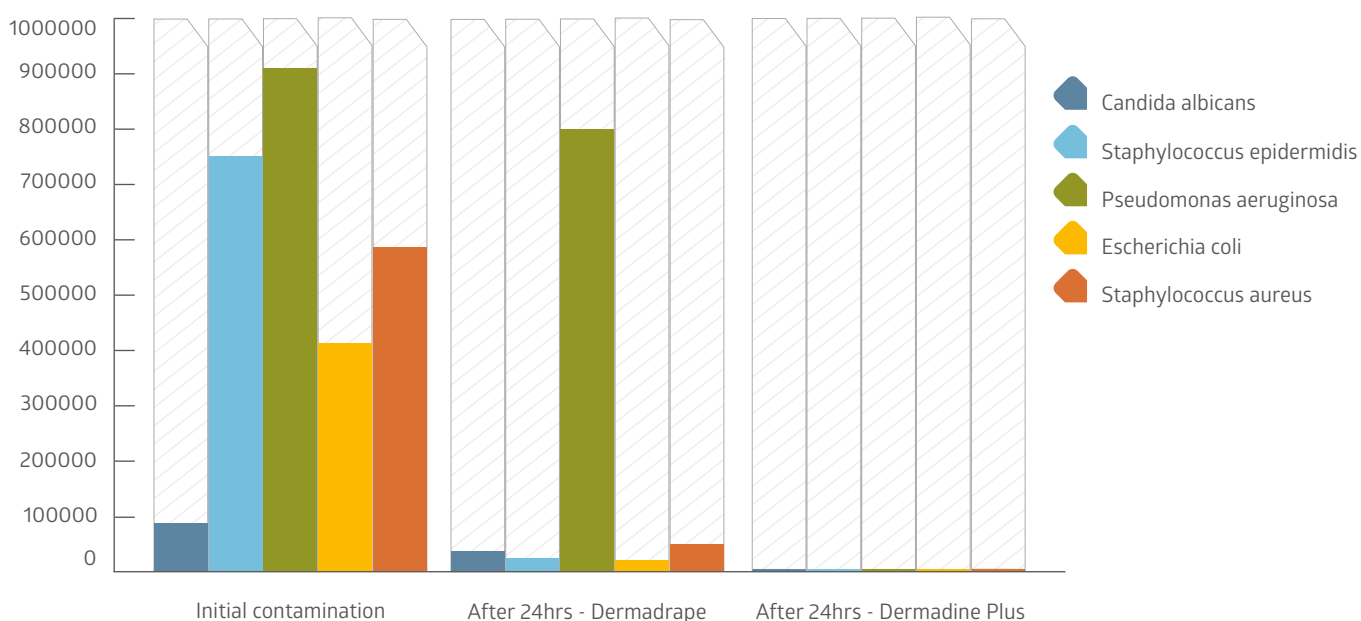




Fast acting

When applied, the Dermadine Plus incise film demonstrates strong antimicrobial activity. Within 30 minutes of application of Dermadine Plus, there is a significant reduction in the bioburden, which gradually stabilises over time. Dermadine Plus guarantees constant asepsis on the patient's skin, over long periods.



Conformable and enveloping

Naturally elastic and ultra thin to guarantee a perfect fit and conformation to anatomical structures and joints. Dermadine Plus adapts to large deformations of muscles (even when autostatic retractors are used) without losing its adhesive properties.

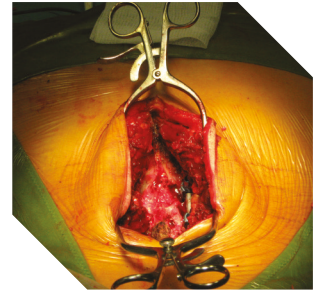

Breathable, water repellent and waterproof

Dermadine Plus combines blood and water repellence with water vapour breathability, to prevent fluid collection that can reduce adhesiveness during long cases. The breathable film guarantees complete impermeability to microorganisms.




Strong and anti-glare

In addition to its excellent resistance to tearing, Dermadine Plus also allows for easy surgical incision with any scalpel blade, and provides indirect protection from tearing for thin skin. Its anti-glare properties provide perfect visibility at the point of incision under the presence of reflected light.



Adhesion with the presence of antimicrobial agent

The skin friendly medical adhesive allows for high peel adhesion on all types of skin surfaces and especially on the edges of the wound without any skin irritation. The poly-acrylic structure in Dermadine Plus adhesive contains iodine.* The polyacrylate complex with the triiodide ion releases iodine molecules after application to the skin. The acrylic adhesive is atraumatic during detachment, even on the skin of elderly patients, and guarantees no residual adhesive after detachment.

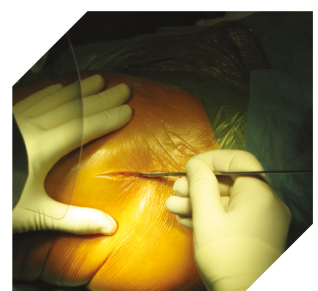

Storage

Store the product at room temperature, protected from light and moisture, in ventilated areas above floor level, in its protective package. Avoid extreme temperatures.

Sterilising agent

Gamma radiation

Disposal

After use, the medical device is considered special waste.












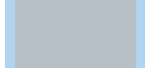

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
Medical device classification

Dermadrape Class IIa
CND code: T02010101
Dermadine Plus Class III
CND code: T02010102

Packaging

- Packed into double-peel open sterilisation envelope
- Removable labels for traceability and patient files
- Available for sale in smaller dispenser boxes
- Available in Medline SPT customised packs



Dermadine Plus item number	Dimensions	Packaging	Dermadrape item number
42020P 	20 x 20 cm Adhesive: 15 x 20 cm	20 ea/bx 120 ea/cs	32020 
n/a	35 x 25 cm Adhesive: 30 x 25 cm	20 ea/bx 120 ea/cs	33525 
44040P 	40 x 40 cm Adhesive: 34 x 40 cm	15 ea/bx 90 ea/cs	34040 
n/a	50 x 48 cm Adhesive: 44 x 48 cm	15 ea/bx 90 ea/cs	35050 
47045P 	70 x 45 cm Adhesive: 60 x 45 cm	15 ea/bx 90 ea/cs	37045 
47060P 	70 x 60 cm Adhesive: 60 x 60 cm	10 ea/bx 60 ea/cs	n/a
47080P 	70 x 80 cm Adhesive: 60 x 80 cm	10 ea/bx 60 ea/cs	n/a
48060P 	80 x 60 cm Adhesive: 60 x 60 cm	10 ea/bx 60 ea/cs	38060 
48080P 	80 x 80 cm Adhesive: 70 x 80 cm	10 ea/bx 60 ea/cs	38080 

*Medline also offers Dermadrape, a full range of incise films without iodine treatment.

For more information on our products, please contact your Medline account manager or visit us at: medline.eu/uk



Medline Industries Ltd.
3rd Floor
Quayside Wilderspool Business Park
Greenalls Avenue
Warrington WA4 6HL
United Kingdom
Tel.: +44 844 334 5237
www.medline.eu/uk
uk-customerservice@medline.com

Medline Ireland Ltd.
1 Grant's Row
Mount Street Lower
Dublin 2
Ireland
Tel.: +353 1 691 73 06

www.medline.eu/ie
ie-customerservice@medline.com

FOLLOW US  