



***STERILE BARRIER SYSTEM
FOR SURGICAL INCISIONS.***

Dermadine® & Dermadrape® Incise Drapes
Increased sterility for a higher safety

DERMADINE® INCISE DRAPE

Cleaning, in preparation for surgery, combined with skin disinfection are two necessary steps for the initial reduction of surface bacteria. However, skin microorganisms are reduced but not eliminated, and can survive in microscopic folds of the skin.

When complete and long-lasting asepsis is required, the Dermadine incise drape guarantees to maintain a sterile field, through its broad-spectrum antimicrobial activity on many microorganisms present on the skin.

The adhesive contains slow-release iodine iodophors which ensure maximum sterility is achieved and sustained over time. It is also suitable for all types of surgical procedure.

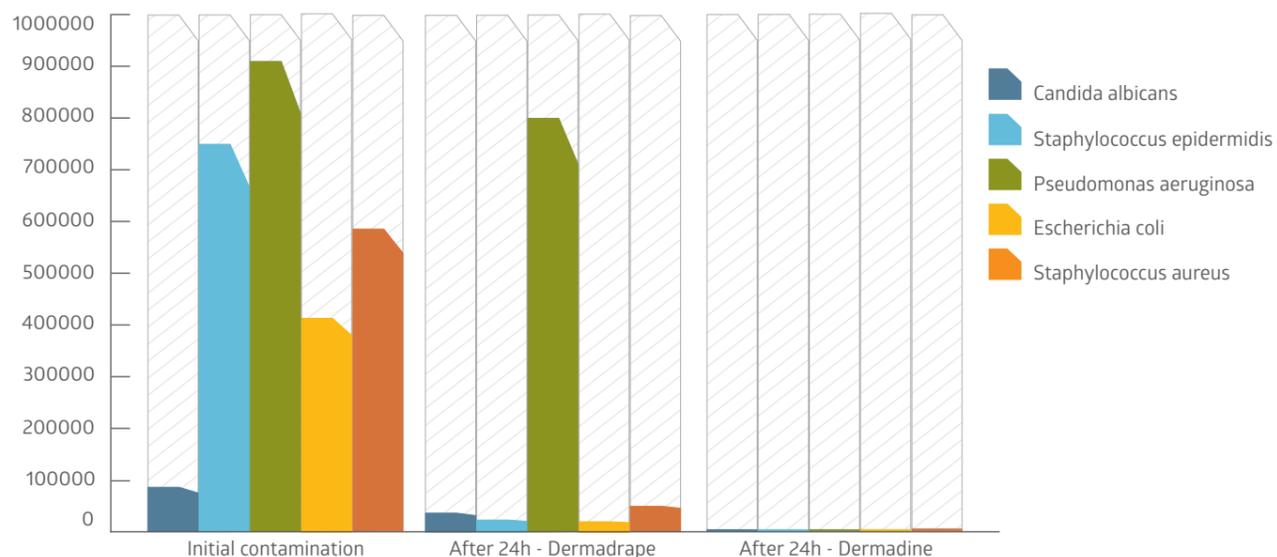
Therefore, Dermadine incise drapes should be used as a final step in preparing the surgical field.

Long lasting

The presence of iodophor groups affixed to acrylates guarantees a release action and a constant presence of iodine for a considerable time, even beyond 24 hours. This characteristic makes the product suitable for long surgical procedures.



MEASUREMENT OF ANTIBACTERIAL ACTIVITY (AS PER ISO 22196:2011)




Fast acting

When applied, the Dermadine incise drape demonstrates strong antimicrobial activity. Within 30 minutes of application of Dermadine, there is a significant reduction in the bioburden, which gradually stabilises over time. Dermadine guarantees constant asepsis on the patient's skin even for long periods of time.


Conformable and enveloping

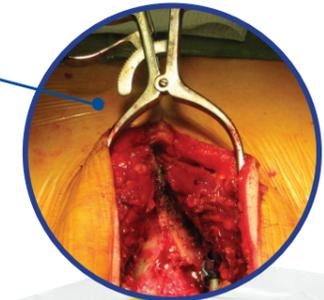Naturally elastic and ultra thin to guarantee a perfect fit and conformation to anatomical structures and joints. Dermadine adapts to large deformations of muscles (even when autostatic retractors are used) without losing its adhesive properties.



Breathable, water repellent and waterproof

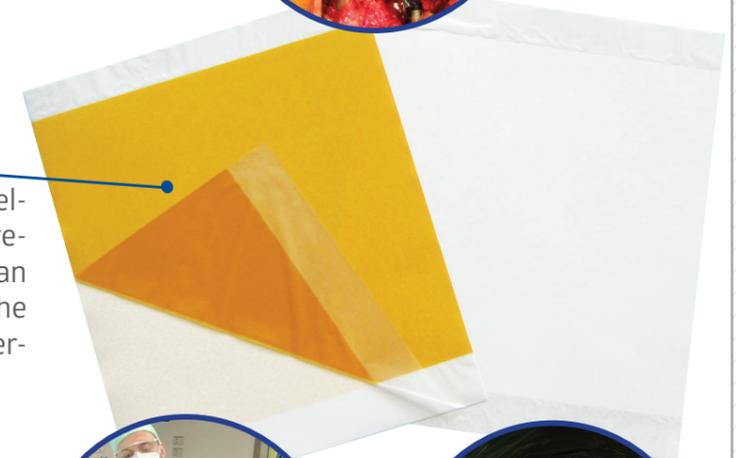Dermadine combines blood and water repellence with water vapour breathability, to prevent the formation of fluid collection that can reduce adhesiveness during long cases. The breathable film guarantees complete impermeability to microorganisms.



Strong and anti-glare

In addition to its excellent resistance to tearing, Dermadine also allows easy surgical incision with any scalpel blade and provides indirect protection from tearing for thin skin.



Its anti-glare properties provide perfect visibility at the point of incision under the presence of reflected light.

Adhesion with the presence of antimicrobial agent

The skin friendly medical adhesive allows high peel adhesion on all types of skin surfaces and especially on the edges of the wound without any skin irritation*. The poly-acrylic structure in Dermadine's adhesive contains an iodophor complex. A surfactant J-carrier, transports iodine and begins releasing it upon application to the skin. The acrylic adhesive is atraumatic during detachment, even on the skin of elderly patients, and guarantees no residual adhesive after detachment.

Storage

Store product at room temperature, protected from light and moisture in ventilated areas above floor level, in its protective package. Avoid extreme temperatures.

Sterilising agent

Gamma radiation

Disposal

After use, the medical device is considered special waste.

Medical device classification

DERMADRAPE®: Class IIa
CND code: T02010101
DERMADINE® Class IIa
CND code: T02010102

Packaging

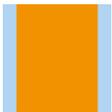
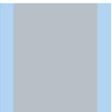
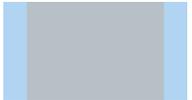
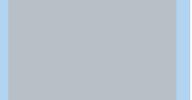
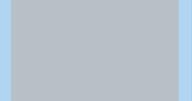
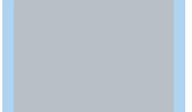
- Packed into individual peelable sterilisation pouch
- Removable labels for traceability and patient files
- Available for sale in smaller dispenser boxes
- Available in Medline SPT customised packs



Tiasset Loc Batriolo
S.P. Km 5,100
52020 Castellfranco di Sopra
(AR) Italia



0373 Istituto Superiore di Sanità

DERMADINE® ITEM NUMBER	DIMENSIONS (CM)	UNITS PER BOX	UNITS PER CASE	DERMADRAPE® ITEM NUMBER*
42020 	20x20cm Adh. 15x20cm	20	120 (6x20)	32020 
NA	35x25cm Adh. 30x25cm	20	120 (6x20)	33525 
44040 	40x40cm Adh. 34x40cm	15	90 (6x15)	34040 
NA	50x48cm Adh. 44x48cm	15	90 (6x15)	35050 
47045 	70x45cm Adh. 60x45cm	15	90 (6x15)	37045 
48060 	80x60cm Adh. 60x60cm	10	60 (6x10)	38060 
48080 	80x80cm Adh. 70x80cm	10	60 (6x10)	38080 

**Medline also offers Dermadrape. A full range of incise films without the iodine treatment.*

For more information on our products, please contact your Medline account manager or visit our site www.medline.eu/uk.



Medline Industries Ltd
No 5 Booths Park
Chelford Road
Knutsford, Cheshire WA16 8GS
England
Tel: +44 844 334 5237
Fax: +44 844 334 5238
www.medline.eu/uk
uk-customerservice@medline.com

Before use, consult instructions for use and precautions on the corresponding labeling.

We reserve the right to correct any errors that may occur within this brochure.

©2017 Medline Industries, Inc. Medline is a registered trademark of Medline Industries, Inc. Three Lakes Drive, Northfield, IL 60093, USA.

ML565-EN02/EB/MM 10/2017