

Rinse-Free Patient Cleansing Products

For improved infection prevention



**Patient
bathing
made easy**



Risks associated with traditional bathing

98%

of basins used for patient hygiene have been found to be contaminated with potentially pathogenic bacteria, including *S. aureus*, VRE, MRSA, *P. aeruginosa*, *C. albicans* and *E. coli*.¹

Controlling the risk of infection in critically ill patients is crucial to their recovery. Patient bathing is one of the key nursing activities that helps improve the quality of life and care for the patient. However, recent research shows it can also be a major source of contamination in intensive care units (ICUs).²

Basins, soap and water can do more than increase the likelihood of spreading infections. Many soaps can be harsh on the skin and can ultimately leave it dry and vulnerable to further damage.



EasyBath® is a better bath

Throw out the bath water

There are many reasons that bedside bathing with EasyBath is superior to traditional basin, soap and water bathing.



Consistent care from cloth to cloth, patient to patient and caregiver to caregiver!

Bathing

EasyBath

EasyBath provides a comforting, consistent and clinically effective bedside bath and is available as impregnated cloths or wash gloves. It helps keep skin feeling healthy because its no-rinse formulation cleans and moisturises without stripping the skin.



Wash cloths

Item code	Description	Packaging
MSCE095309A	Standard; fragrance-free	8 cloths/pack; 30 packs/case

Wash gloves



Item code	Description	Packaging
WM80300	Standard; scented	8 gloves/pack; 30 packs/case
WM80301	Standard; fragrance-free	8 gloves/pack; 30 packs/case
WM50303	Standard; fragrance-free	5 gloves/pack; 24 packs/case



EasyBath Plus

EasyBath Plus Antiseptic Patient Wash Gloves and Wash Cloths are specially created for antiseptic patient bathing, as they are impregnated with a 2 per cent chlorhexidine digluconate (CHG) solution. Our impregnated wash gloves and wash cloths can be used before and after surgical procedures, even on high-risk patients, as the 2 per cent CHG will act as a skin disinfectant and help with reducing the microbial load on the skin.



Wash cloths

Item code	Description	Packaging
CHGC800	Standard; impregnated with CHG	8 cloths/pack; 30 packs/case



Wash gloves

Item code	Description	Packaging
CHGWM800	Standard; impregnated with CHG	8 gloves/pack; 24 packs/case

Haircare

Shampoo caps

Bed-bound shampooing is a challenge. But without the spills and hassles of basins, suds, rinsing and repositioning, EasyBath scented and unscented shampoo caps make clean hair a pleasure for everyone involved. EasyBath shampoo caps clean and condition hair, leaving it fresh and ready for styling. And they also enable healthcare providers to give a soothing, relaxing and gentle scalp massage.



Item code	Description	Packaging
SC230	Standard; scented	1 cap/pack; 30 packs/case
SC231	Standard; fragrance-free	1 cap/pack; 30 packs/case



Patient comfort

Intelligent Warmer

For the ultimate patient and resident comfort, the exclusive EasyBath Intelligent Warmer keeps EasyBath packs at an even, safe temperature for a nice warm bath. Indicator lights glow when the ideal temperature has been reached. It even helps track inventory by indicating the freshest pack to take, maintaining a first in, first out (FIFO) stock rotation.



Item code	Slots	Size	Weight
MSCEWARMER14S	14	33 x 46 x 34 cm	10.6 kg
MSCEWARMER28S	28	56 x 46 x 34 cm	12.9 kg

Personal clean-up

AloeTouch®

The Flip 'N Seal™ closure on AloeTouch packs reduces the amount of plastic in the lid by 71 per cent while keeping the wipes fresh for weeks. Unlike traditional adhesive film closures, Flip 'N Seal opens easily with one hand and stays open until you close it.

AloeTouch is the choice for people who want a little more from their cleansing wipe. The gentle, sulphate-free cleansing solution and aloe-rich skin conditioners permeate this heavier-weight wipe. You may find that you use fewer wipes to get the job done.



Item code	Description	Packaging
MSCE263654	Spunlace wipes; scented , 20 x 30 cm	48 wipes/pack; 12 packs/case

ReadyFlush®

To be labelled 'flushable' by EDANA,* ReadyFlush wipes had to pass seven tests that evaluated their ability to disintegrate, biodegrade and clear septic systems and sewer lines. That means they break down just like toilet paper when flushed for a no-clog performance. They are 100 per cent biodegradable and safe for sewers and septic systems.

ReadyFlush is at home by the bedside or right there in the bathroom. It's strong enough for vigorous cleansing, while its mild formula is soothing on the skin.



Item code	Description	Packaging
PW840	Flushable wipes; scented , 20 x 25 cm	48 wipes/pack, 12 packs/case

*EDANA: European Disposables and Nonwovens Association

Standardise your bathing protocol now!

3 easy steps:

MEDLINE ALGETOUCH® PRODUCT TEST
EVALUATION FORM

Name: _____ Date: _____

Which products are you currently using to wash your patients? _____

HOW SATISFIED ARE YOU WITH THE TESTED PRODUCT?

	Excellent	Very Good	Acceptable	Unsatisfactory	N/A
Packaging and removal from packaging	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Material and structure of the product	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Performance	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Ease of use	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Time expenditure	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Cleaning of patient	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Tissue condition	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Overall impression	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

If you have other comments, please share them in the comment field below:
Comments: _____

Signature: _____

Your opinion is important to us! Thank you for taking the time to test and evaluate our products.

MEDLINE

How to Use: EasyBath® Shampoo Cap

Convenient and quick haircare

- 1 Prepare a towel, wash your hands and put on examination gloves if needed.
- 2 Warm the EasyBath Shampoo Cap for 15 seconds in the microwave (250 watts maximum) or in Medline's Intelligent Warmer. Tear open the packaging and remove the cap.
- 3 Place the cap on the patient's head. If the patient has longer hair, you will need to gather the hair on top of the head prior to putting the cap on.
- 4 Massage the patient's hair through the cap until the hair is saturated. It is 3 minutes for shorter hair and 2 to 3 minutes for longer hair.
- 5 Remove the cap and discard it in the appropriate waste receptacle. No rinsing is required.
- 6 Use a towel to dry the patient's hair if necessary and then comb through the hair.

For more information about EasyBath, contact your Medline account manager or visit www.medline.eu/uk

EASYBATH
BATHING WITHOUT WATER

1 Evaluation of customer needs with Medline clinical staff

2 Education of nurses

3 Education of patient and family

For more information, contact your Medline account manager or visit us at: www.medline.eu/uk.



Medline Industries Ltd.
3rd Floor
Quayside Wilderspool Business Park
Greenalls Avenue
Warrington WA4 6HL
United Kingdom
Tel.: +44 844 334 5237
www.medline.eu/uk
uk-customerservice@medline.com

Medline Ireland Ltd.
1 Grant's Row
Mount Street Lower
Dublin 2
Ireland
Tel.: +353 1 691 73 06
www.medline.eu/ie
ie-customerservice@medline.com

FOLLOW US

¹Johnson D, Lineweaver L, Maze LM. Patients' bath basins as potential sources of infection: a multicenter sampling study. *American Journal of Critical Care*. 2009; 18(1):31-38.
²Decker BK, Palmore TN. The role of water in healthcare-associated infections. *Current Opinion in Infectious Diseases*. 2013;26(4):345-351. doi:10.1097/QCO.0b013e3283630adf
³Kron-Chalupa J, Benda T, Williams B. The basinless bath: A study on skin dryness and patient satisfaction. *Iowa City Veterans Affairs Medical Center*. 2006.