

Rinse-free patient cleansing products

For improved infection prevention



**Patient
bathing
made easy**



Risks of traditional bathing

98%

of basins used for patient hygiene are contaminated with potentially pathogenic bacteria, including *S. aureus*, VRE, MRSA, *P. aeruginosa*, *C. albicans* and *E. coli*.¹

Controlling risks of infection in critically ill patients is crucial for their recovery. Patient bathing is one of the key nursing activities that help improve the quality of life and care for the patient. However, recent research shows it can also be a major source of contamination in intensive care units (ICUs).²

Basins, soap and water can do more than increase the likelihood of spreading infections. Many soaps can be harsh on the skin and can ultimately leave it dry and vulnerable to further damage.



EasyBath® is a better bath

Throw out the bath water

There are many reasons that bedside bathing with EasyBath is superior to traditional basin, soap and water bathing.



Consistent care from cloth to cloth, patient to patient and caregiver to caregiver!

Bathing

EasyBath

EasyBath provides a comforting, consistent and clinically effective bedside bath and is available as impregnated cloths or wash mitts. It helps keep skin feeling healthy because its no-rinse formulation cleans and moisturises without stripping the skin.



Wash cloths

Item code	Description	Packaging
MSCE095309A	Standard; fragrance free	8 wipes/pack; 30 packs/case

Wash gloves



Item code	Description	Packaging
WM80300	Standard; scented	8 mitts/pack; 30 packs/case
WM80301	Standard; fragrance free	8 mitts/pack; 30 packs/case
MSCE995302WM	Standard; scented	5 mitts/pack; 24 packs/case



Non-woven dry wash gloves

Item code	Description	Packaging
MEDGLOVE01	16.5 x 23 cm, 75 GSM	100 gloves/pack; 20 packs/case
MEDGLOVE02	14.5 x 20 cm, 60 GSM	100 gloves/pack; 20 packs/case
MEDGLOVE03	14.5 x 20 cm, 75 GSM	100 gloves/pack; 20 packs/case



Dry wash gloves are the perfect complement to the EasyBath range, providing a cost-effective solution that is convenient and comfortable for the patient.

Hair care

Shampoo caps

Bed-bound shampooing is a challenge. But without the spills and hassles of basins, suds, rinsing and repositioning, EasyBath scented and unscented shampoo caps make clean hair a pleasure for everyone involved. EasyBath shampoo caps clean and condition hair, leaving it fresh and ready for styling. And they also enable healthcare providers to give a soothing, relaxing and gentle scalp massage.



Item code	Description	Packaging
SC230	Standard; scented	1 cap/pack; 30 packs/case
SC231	Standard; fragrance free	1 cap/pack; 30 packs/case



Patient comfort

Intelligent Warmer

For the ultimate patient and resident comfort, the exclusive EasyBath Intelligent Warmer keeps EasyBath packs at an even, safe temperature for a nice warm bath. Indicator lights glow when the ideal temperature has been reached. It even helps track inventory by indicating the freshest pack to take, maintaining a FIFO stock rotation.



Item code	Slots	Size	Weight
MSCEWARMER14S	14	33 x 46 x 34 cm	10.6 kg
MSCEWARMER28S	28	56 x 46 x 34 cm	12.9 kg

Personal clean-up

AloeTouch®

The Flip 'N Seal™ closure on AloeTouch packs reduces the amount of plastic in the lid by 71 per cent while keeping the wipes fresh for weeks. Unlike traditional adhesive film closures, Flip 'N Seal opens easily with one hand and stays open until you close it.

AloeTouch is the choice for people who want a little more from their cleansing wipe. The gentle, sulphate-free cleansing solution and aloe-rich skin conditioners permeate this heavier-weight wipe. You may find that you use fewer wipes to get the job done.




Item code	Description	Packaging
MSCE263654	Spunlace wipes; scented , 20 x 30 cm	48 wipes/pack; 12 packs/case

ReadyFlush®

To be labelled 'flushable' by EDANA,* ReadyFlush wipes had to pass seven tests that evaluated their ability to disintegrate, biodegrade and clear septic systems and sewer lines. That means they break down just like toilet paper when flushed for a no-clog performance. They are 98 per cent biodegradable and safe for sewers and septic systems.

ReadyFlush is at home by the bedside or right there in the bathroom. It's strong enough for vigorous cleaning, while its mild formula is soothing on the skin.

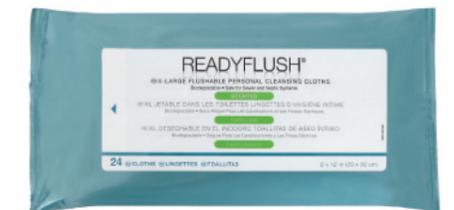

Item code	Description	Packaging
MSCE263810	Flushable wipes; scented , 20 x 30 cm	24 wipes/pack; 24 packs/case

*EDANA: European Disposables and Nonwovens Association

Standardise your bathing protocol now!

3 easy steps:

MEDLINE ALGETOUCH® PRODUCT TEST
EVALUATION FORM

Name: _____ Date: _____

Which products are you currently using to wash your patients? _____

HOW SATISFIED ARE YOU WITH THE TESTED PRODUCT?

	Excellent	Very Good	Acceptable	Unsatisfactory	N/A
Advantages					
Prepared and removed from packaging	<input type="checkbox"/>				
Material and structure of the product	<input type="checkbox"/>				
Performance	<input type="checkbox"/>				
Ease of use	<input type="checkbox"/>				
Time expenditure	<input type="checkbox"/>				
Cleaning of patient	<input type="checkbox"/>				
Patients satisfaction	<input type="checkbox"/>				
Overall impression	<input type="checkbox"/>				

If you have other comments, please share them in the comment field below:
Comments: _____

Signature: _____

Your opinion is important to us! Thank you for taking the time to test and evaluate our products.

MEDLINE

How to Use: EasyBath® Shampoo Cap

Convenient and quick haircare

- 1 Prepare a towel, wash your hands and put on examination gloves if needed.
- 2 Warm the EasyBath Shampoo Cap for 15 seconds in the microwave (250 watts maximum) or in Medline's Intelligent Warmer. Tear open the packaging and remove the cap.
- 3 Place the cap on the patient's head. If the patient has longer hair, you will need to gather the hair on top of the head prior to putting the cap on.
- 4 Massage the patient's hair through the cap until the hair is saturated. It is 2 minutes for shorter hair and 2 to 3 minutes for longer hair.
- 5 Remove the cap and discard it in the appropriate waste receptacle. No rinsing is required.
- 6 Use a towel to dry the patient's hair if necessary and then comb through the hair.

MEDLINE

For more information about EasyBath, contact your Medical Account Manager or visit www.medline.eu/uk

EASYBATH
BATHING WITHOUT WATER

1 Evaluation of customer needs with Medline clinical staff

2 Education of nurses

3 Education of patient and family

For more information, contact your Medline account manager or visit us at: uk.medline.eu.



Medline Industries Ltd
3rd Floor
Quayside Wilderspool Business Park
Greenalls Avenue
Warrington WA4 6HL
United Kingdom
Tel.: +44 844 334 5237
Fax: +44 844 334 5238
uk.medline.eu | uk-customerservice@medline.com

FOLLOW US

¹Johnson D, Lineweaver L, Maze LM. Patients' bath basins as potential sources of infection: a multicenter sampling study. Am J Crit Care. 2009. 18(1):31-38.
²Decker BK, Palmore TN. The role of water in healthcare-associated infections. Curr Opin Infect Dis. 2013;26(4):345-351. doi:10.1097/QCO.0b013e3283630adf
³Kron-Chalupa J, Benda T, Williams B. The basinless bath: A study on skin dryness and patient satisfaction. Iowa City Vettrans Affairs Medical Center. 2006.