





HOW TO USE: EASYBATH 5-PACK BATHING SYSTEM

Warm the EasyBath pack (closed) in the Medline Intelligent Warmer or in the microwave for 30 to 60 seconds (750-watt setting).

Use one EasyBath wash glove to cleanse each of the following five areas of the body:

- 1 Face, neck and chest
- 2 Arms and armpits
- 3 Perineum
- 4 Legs
- 5 Back/buttocks.

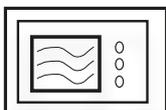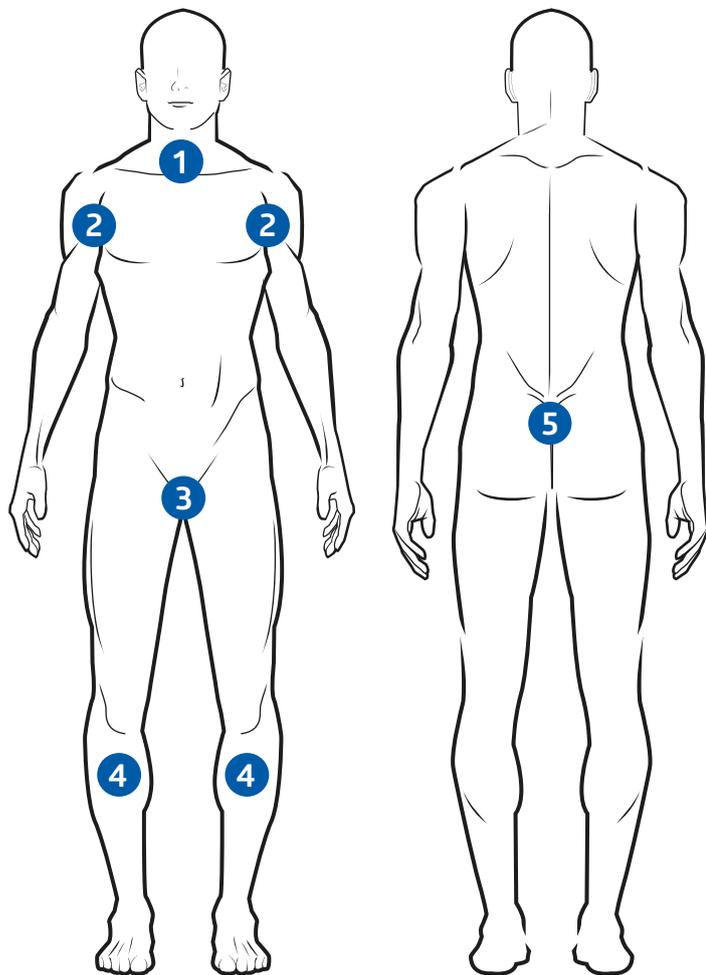

Microwavable
750 watts
30 - 60 seconds



Do not flush EasyBath
wash gloves down the toilet.

For more information about EasyBath, contact your Medline account manager or visit: www.medline.eu