

EasyBath® 5-Pack Bathing System




How to use:

Warm the EasyBath pack (closed) in the Medline EasyBath Intelligent Warmer.

Use one EasyBath wash glove to cleanse each of the following five areas of the body:

- 1 Face, neck and chest
- 2 Arms and armpits
- 3 Perineum
- 4 Legs
- 5 Back/buttocks.

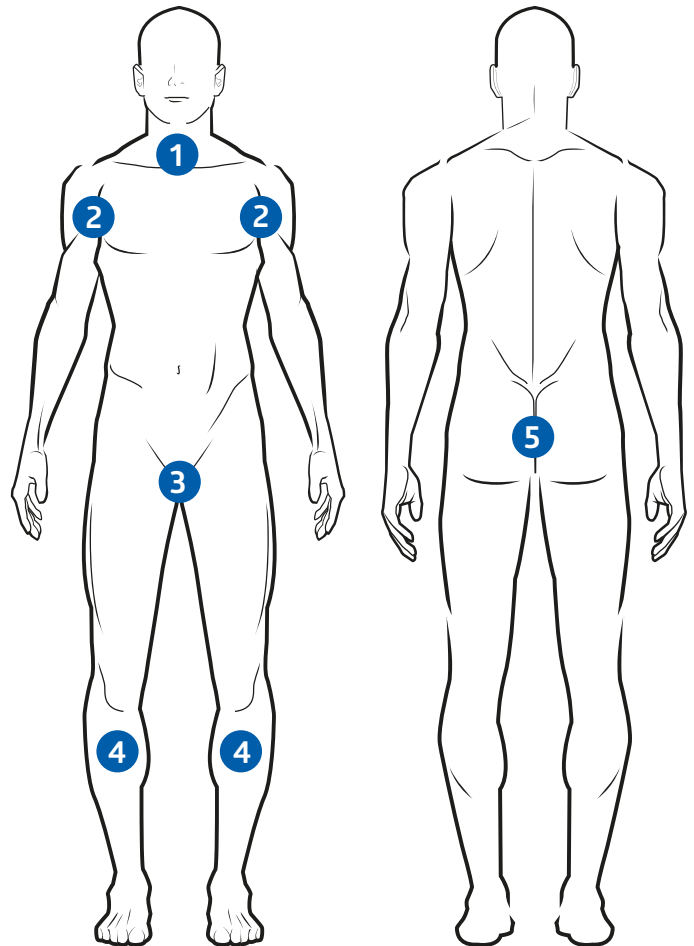

WM50303

For more information about EasyBath, contact your Medline account manager or visit: www.medline.eu/uk.



Do not flush EasyBath washcloths or wash gloves down the toilet.

*For exact heating instructions, please refer to the corresponding labels and instructions for use.

© 2023 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP. ML199_EN 09/2023