

EasyBath® 8-Pack Bathing System




How to use:

Warm the EasyBath pack (closed) in the Medline Intelligent Warmer or in the microwave (750-watt setting).

Use one EasyBath wash glove or washcloth to cleanse each of the following eight areas of the body:

- 1 Face, neck and chest
- 2 Left arm/armpit
- 3 Right arm/armpit
- 4 Perineum
- 5 Left leg
- 6 Right leg
- 7 Back
- 8 Buttocks.

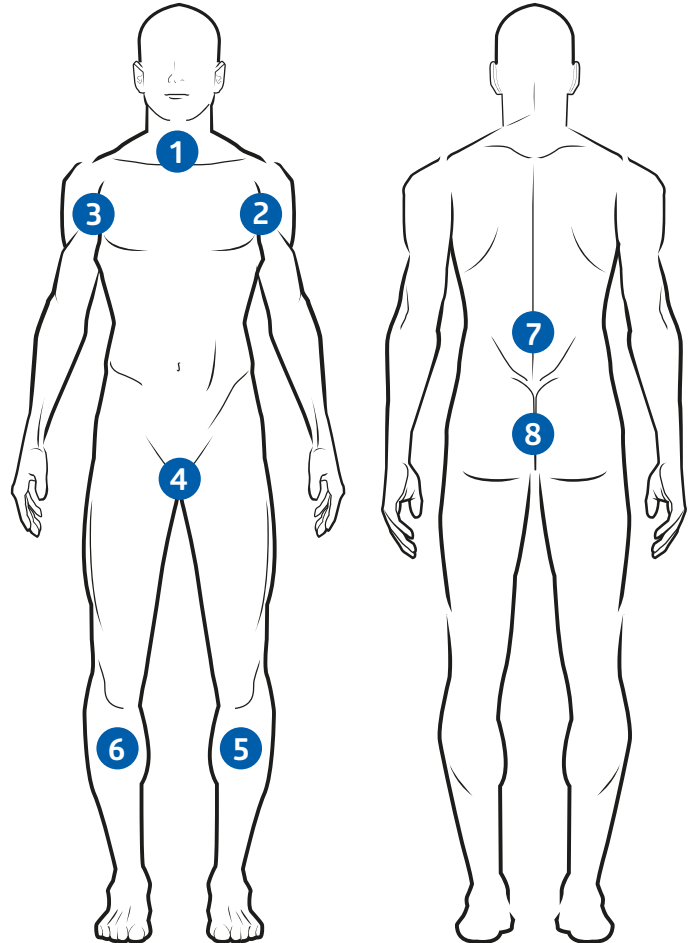

WM80300/WM80301



MSCE095309A



Microwaveable
750 watts
30 seconds*



Do not flush EasyBath
washcloths or wash
gloves down the toilet.

For more information about EasyBath, contact your Medline account manager or visit: www.medline.eu/uk.

*For exact heating instructions, please refer to the corresponding labels and instructions for use.