



# How to Use: EasyBath® Shampoo Cap

## Convenient and quick haircare

- 1 Prepare a towel, wash your hands and put on examination gloves if needed.
- 2 Warm the EasyBath Shampoo Cap for 15 seconds in the microwave (1250 watts maximum) or in Medline's Intelligent Warmer. Tear open the packaging and remove the cap.
- 3 Place the cap on the patient's head. If the patient has longer hair, you will need to gather the hair on top of the head prior to putting the cap on.
- 4 Massage the patient's hair through the cap until the hair is saturated: 1 to 2 minutes for shorter hair and 2 to 3 minutes for longer hair.
- 5 Remove the cap and discard it in the appropriate waste receptacle. No rinsing is required.
- 6 Use a towel to dry the patient's hair if necessary and then comb through the hair.



MICROWAVEABLE  
Max. 1250 W  
15 seconds



SC230: scented



SC231: fragrance-free



For more information about EasyBath, contact your Medline account manager or visit: [www.medline.eu/uk](http://www.medline.eu/uk)