

Endocavity Probe Covers



A protective solution for transrectal and transvaginal ultrasound procedures

The Medline Endocavity Probe Cover serves as a viral barrier for transrectal and transvaginal ultrasound procedures. It is compatible with most ultrasounds.

This cover offers maximum protection against cross-contamination to both your patient and equipment. It also helps to reduce the time required to remove gel, excrement and other debris to which the probe may ordinarily be exposed during invasive medical examinations.

Uses

Each Medline Endocavity Probe Cover can be used during gynaecological, urological and proctological interventions.

Benefits and features



Compatible with most ultrasound transducers



Sterile and non-sterile ultrasound probe covers available



Transparent to promote easy visibility and clear imagery



Lubricant option available to ensure optimal custom fitting



Effective barrier with latex and latex-free (polyisoprene) option to help prevent allergic reactions in patients and/or healthcare professionals; the material is soft and highly resistant.




These covers fit most transducers perfectly. They are available in both latex and non-latex materials, as well as sterile or non-sterile, to help ensure optimum custom fitting.




Instructions for application

1

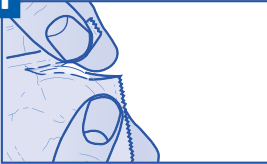


Gently tear open the package along the edges or corners. Handle with care to avoid puncturing the endocavity probe covers.

2



Apply the cover over the transducer. Be careful to not leave bubbles or creases that could interfere with proper visibility and imagery. You can also apply gel on the transducer if necessary.

3



Once the transducer is fully covered, inspect the instrument one last time, to make sure there are no bubbles or openings that might make the cover less effective.

Ordering information

Latex endocavity probe covers

Item number	Dimensions	Packaging	Sterile
NS-ICE28295L	28 x 295 mm	144 ea/bx; 2880 ea/cs	No
NS-ICE33230L	33 x 195 mm	144 ea/bx; 2880 ea/cs	No
NS-ICE33230LL*	33 x 195 mm	144 ea/bx; 2880 ea/cs	No
ICE28295L	28 x 295 mm	100 ea/bx; 2000 ea/cs	Yes
ICE33230L	33 x 195 mm	100 ea/bx; 2000 ea/cs	Yes

*With lubricant

Polyisoprene endocavity probe covers

Item number	Dimensions	Packaging	Sterile
NS-ICE28295P	28 x 295 mm	100 ea/bx; 2000 ea/cs	No
ICE28295P	28 x 295 mm	100 ea/bx; 2000 ea/cs	Yes

Learn more. Contact your Medline account manager or visit: www.medline.eu/uk



Medline Industries Ltd.
3rd Floor
Quayside Wilderspool Business Park
Greenalls Avenue
Warrington WA4 6HL
United Kingdom
Tel.: +44 844 334 5237
www.medline.eu/uk
uk-customerservice@medline.com

Medline Ireland Ltd.
1 Grant's Row
Mount Street Lower
Dublin 2
Ireland
Tel.: +353 1 691 73 06

www.medline.eu/ie
ie-customerservice@medline.com

FOLLOW US  

These products are class IIa medical devices intended to be used by healthcare professionals. Before use, consult instructions and precautions on the corresponding labelling.

We reserve the right to correct errors that may occur within this brochure.

© 2023 Medline is a registered trademark of Medline Industries, LP. Three Lakes Drive, Northfield, IL 60093, USA. ML1370_EN 07/2023

