



Surgical Hand Brush.


Single Use Surgical Hand Brush Impregnated with Povidone-iodine (7.5%)
 User Manual. **REF 205 02**

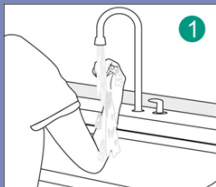


1 Wet hands, wrists and forearms to the elbows with water. Hands should be held higher than the elbows at all times

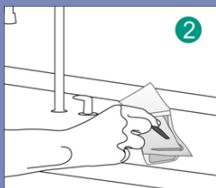

2 Clean under nails with nail pick provided

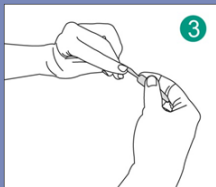

3

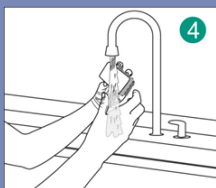


4 Use the brush side to clean the nails, cuticles and interdigital spaces between the fingers and the sponge side to scrub the hands and forearms. Wet sponge and squeeze to work up lather



5



8*

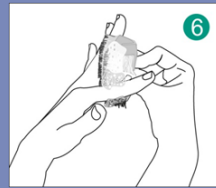

6



9*



7*


Scrub the nails, cuticles and interdigital spaces between the fingers, hands and forearms for 3 minutes in total

* Scrub for an additional 4 minutes with using the sponge side only




10

Rinse the hands, wrists and forearms with plenty of water

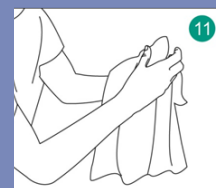

11

Dry with a sterile towel from hands to elbows. Hold hands upwards

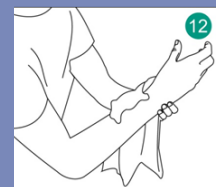

12

medbar[®]