



# HEELMEDIX<sup>®</sup>

All-in-one pressure relief and skin protection.



Every heel needs protection.



Treated with Ultra-Fresh® antimicrobial treatment which helps to inhibit odour-causing bacteria.<sup>2</sup>



Equipped with 3-hole options on each side of the boot for DVT pressure devices.

## RELIEVE PRESSURE AND PROTECT VULNERABLE HEELS.

### Clinical Studies Show Effective Results<sup>1</sup>

Using a pressure-sensing mat, 19 patients in a long-term care facility were tested for heel pressure on a mattress and in a Heelmedix device. Heelmedix completely offloaded the vulnerable heel area. Here are the pressure mapping conclusions:




Red dotted line indicates heel



Average pressure on contact area **115.23**  
Mattress had the highest heel pressure.



Average pressure on contact area **29.82**  
Heelmedix completely offloaded the vulnerable heel area.

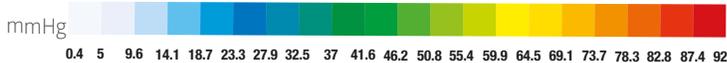

Item No.	Description	Pkg
MDTE823330P	Heelmedix, Petite, Heel Protector	1/ea
MDTE823330SW	Heelmedix, w/ Wedge, Heel Protector	1/ea
MDTE823330XL	Heelmedix, XL, w/Wedge, Heel Protector	1/ea

### BENEFITS

- » Completely offloads the heel and helps redistribute pressure.
- » Screen printing of “toes” and “heel” eliminates confusion to ensure the boot is applied properly.
- » Open heel design allows for airflow and easy monitoring.
- » Gray straps pull down on sides to help protect against foot drop.
- » An optional wedge can be used to help prevent medial & lateral rotation.
- » Patented side hole design minimizes chance of SCD tubing touching patient’s skin.
- » Inner lining helps reduce friction and shear - half the friction of the competitor.<sup>1</sup>
- » Includes a mesh laundry bag to simplify washing and sorting.



ALWAYS ON.

Medline Industries Ltd  
3rd Floor  
Quayside Wilderspool Business Park  
Greenalls Avenue  
Warrington WA4 6HL  
United Kingdom  
Tel.: +44 844 334 5237  
Fax: +44 844 334 5238

[www.medline.eu/uk](http://www.medline.eu/uk)  
[uk-customerservice@medline.com](mailto:uk-customerservice@medline.com)

REFERENCES 1. Data on file and available upon request. 2. Ultra-Fresh is an EPA-registered antimicrobial. This product does not protect users or others from disease-causing bacteria.

© 2018 Medline Industries, Inc. All rights reserved. HEELMEDIX and Medline are registered trademarks of Medline Industries, Inc. Ultra-Fresh is a registered trademark of Thomson Research Associates. ML268-EN 11/2018.