

How to apply: Procedure Masks

Recommended to change mask every 3 hours



1 Wash/clean hands



2 Ensure Medline stamp faces outwards and the conformable band is uppermost



3 Place loops behind ears and cover with headwear



4 Conform band to the bridge of your nose



5 Ensure pleats are fully extended to cover nose and mouth



6 If there is risk of splashing, use appropriate eye protection



7 After use, remove and dispose of mask immediately



8 Dispose of the mask correctly



9 Wash your hands after disposing of the mask

To learn more about procedure masks, contact your Medline account manager or visit www.medline.eu/uk

