

How to apply: Respirator Masks

All respirators should be FIT tested in accordance with local policy. Change mask if it becomes contaminated or every 8 hours.



1 Wash/clean hands



2 Ensure the metal band is up and straps hang below



3 Hold the respirator over your mouth and nose



4 Place upper strap at crown of head



5 Place lower strap at base of neck



6 Conform metal nose clip to the bridge of your nose



7 Cup both hands over respirator without disturbing position and inhale vigorously. If air leaks around the edges, reposition for better fit.



8 If there is risk of splashing, use appropriate eye protection



9 After use, remove mask and dispose of correctly



10 Wash your hands after disposing of the mask

To learn more about respirator masks, contact your Medline account manager or visit www.medline.eu/uk