



MED-SOFT

Suction Restart

Please find the instructions below for restarting suction after premature shut-off of the MED-SOFT Fluid Management System. This means that the vacuum is ON, but the suction does not work.


1 Check that the canister is in a fixed, vertical position.

2 With the vacuum ON, disconnect the red vacuum tube from the vacuum port (the shut-off valve will move back into the open position).

3 Reconnect the red vacuum tube to the vacuum port, and then the suction will start again.

WHY IS THIS FEATURE IN PLACE?

The vacuum port is protected by a shut-off valve and an antibacterial filter. These two simple and efficient features prevent the liquid from flowing back into the suction source of your healthcare facility.

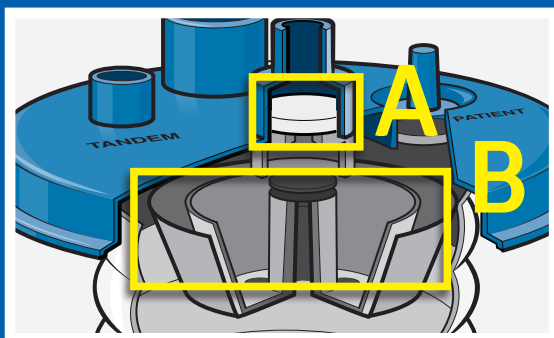

- A. Antibacterial filter against aerosols
- B. Shut-off valve to shut off the suction if the canister is bumped or suddenly moved out of its vertical position

 Always place soft liners in a fixed, reusable outer canister carrier.

For more information about Medline suction systems, please contact your account manager or visit us at: www.medline.eu/uk