



MED-SOFT

Liner Insertion

Please find the instructions below for properly inserting the MED-SOFT liner into the canister.



Always attach the outer reusable canister to a holder before inserting the soft liner so that the canister doesn't fall. Make sure to always use a canister and liner with the same capacity.

1

Remove the liner from its packaging via the easy opening.



Extend the liner by pulling the top and bottom away from each other.



Straighten it out.



2

To facilitate the liner's insertion into the canister, close the TANDEM port (with the cap or your finger) to keep air in the liner.



3

Press down on the liner lid to ensure that the liner is securely sealed to the outer canister.



When the vacuum is ON, it helps seal the liner lid to the canister.

4

Check the vacuum connection beneath the red bracket. Attach the red tubing from the vacuum source to the vacuum port on the soft liner lid. Turn the vacuum ON.



5

Put your finger on the L-connector to securely seal the liner to the outer canister with the vacuum.



6

Connect the patient tubing on the L-connector to the patient port.



Remember to deactivate the vacuum when you don't require suction.

For more information about Medline suction systems, please contact your account manager or visit us at: www.medline.eu/uk