



MED-SOFT Liner Removal

Please find the instructions below for properly removing the MED-SOFT liner from the canister.

- 1** Disconnect the suction connective tubing from the patient port. Close the port with the cap.



Remove the elbow connector with the patient tubing.

- 2** Ensure that all ports are capped off and sealed, besides the vacuum port.



If you are using the tandem tube, disconnect it from the patient port and close the cap.

- 3** If you want to add a gelling agent, open the accessory port, insert the correct quantity of the gelling agent and close the port.



- 4** Disconnect the red vacuum tube from the vacuum port and cap that port securely.



When disconnecting the vacuum tube, the liner can jump out of its canister. This is normal.

- 5** Turn the vacuum OFF.



- 6** Remove the suction soft liner from the outer canister. Make sure to comply with the facility's protocols and the local state and federal requirements when disposing of soft liners and their contents.



For more information about Medline suction systems, please contact your account manager or visit us at: www.medline.eu/uk