



## HOW TO USE THE SURGICAL CLIPPER



### 1 Attaching blades

- Align the mark on the blade to the top line on the clipper.
- Slide the blade down until it aligns with the lower line on the clipper.
- A 'click' is heard.



### 4 Removing blades

- Make sure the clipper is in the 'OFF' position.
- Place your thumb on the textured indentation on the back of the blade.
- Push until the blade pops off.



### 2 Holding the clipper

- For proper use, hold the clipper like a pen.
- Non-slip handle side grips allow for full control of the clipper and one-handed manoeuvrability.



### 5 Cleaning and disinfecting

- Wipe it down with non-corrosive universal wipes between each patient use.
- Clean it by rinsing with water as soon as possible after each use.



### 3 Clipping on wet or dry skin

- Make sure the clipper is in the 'ON' position.
- Use short gentle strokes to remove the hair.
- For best results, the direction of the movement while clipping should go against the grain.



### 6 Storing the clipper

- Make sure the clipper is in the 'OFF' position.
- Place it on the charging station.

These products are Class I medical devices manufactured by Medline industries Inc.  
For more information, please refer to the IFU and packaging.



For more information, please contact your account manager or visit our website: [www.medline.eu/uk](http://www.medline.eu/uk).