

Preparation for use: NONELENS and NONEFRAME




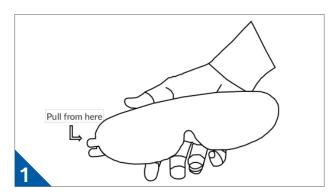


Remove the lens (NONELENS) from the dispenser.

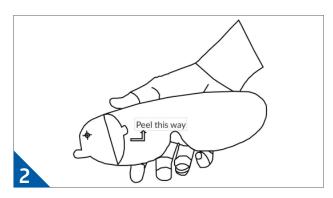


Peel away the protective inner film by pulling the upper tab towards you.

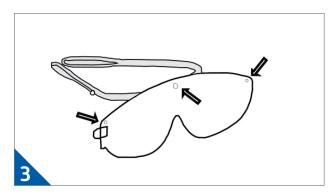

Clip three holes into the corresponding reusable frame (NONEFRAME). The remaining film should be on the outside.

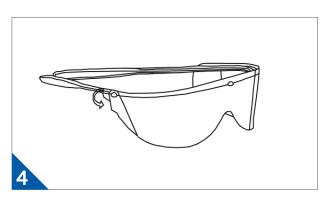

Remove the outer peel using the same method in Step 2.

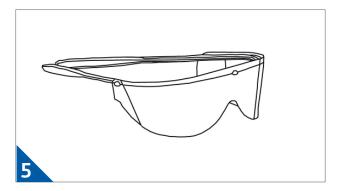

Your device can now be worn and is free from scratches.

Note: To clean or disinfect your NONEFRAME, wipe it gently using liquid with up to 70% Isopropanol.

To learn more about procedure masks, contact your Medline account manager or visit uk.medline.eu

