



EasyBath®

Intelligent Warmer


Model MSCEWARMER14S and MSCEWARMER28S

- 1** Place the Intelligent Warmer in the location where it will be operated and make sure the air vents are not obstructed. Plug the power cord into the AC power module and turn on the device using the power switch located on the back of the warmer.
- 2** The Intelligent Warmer is ready for immediate use. The LEDs on the upright sensor panels will illuminate, first red and then green, and will self-test until the current chamber temperature is displayed. The warmer will begin heating up to the factory preset temperature of 52° Celsius.
- 3** If you wish to change the preset temperature, you can do so with the following steps. Press the 'P' button once and use the arrows to adjust the temperature. To return to the original display, press 'P' until you see the warmer's temperature. The Intelligent Warmer operates in the range of 38° to 54° Celsius.
- 4** Load the EasyBath products. It is normal for the temperature on the display to drop after adding new products into the chamber. As each EasyBath product is added, status indicators will illuminate. The Intelligent Warmer remembers the sequence in which the products were added.

Light indicators

- Solid red light: warming
- Solid green light: heated and ready to use
- Flashing green light: heated, take out first
- Flashing red light: discard, product has been in the warmer longer than 250 hours



 For more detailed instructions, please see the device manual included in the packaging.