


Procedure Guide: ZR[®] Pre-filled Flush Syringe

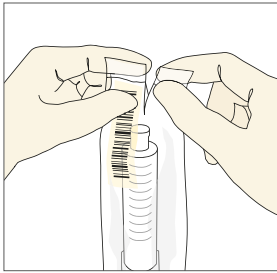


1

- Inspect package integrity (wrapping, syringe and tip cap).

Note: Do not use if the packaging has been damaged, the syringe is leaking or tip cap is not fully attached to the syringe.

- Remove protective wrapping
- Visually inspect the syringe and solution


Note: Do not use if either is cloudy or hazy.

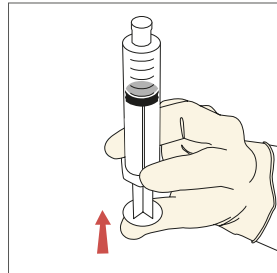

2

Before removing the cap:

- Hold the syringe with the tip cap upright.
- Press the syringe plunger until the plunger moves slightly. This motion breaks the seal between the syringe and allows for smooth application of the solution.

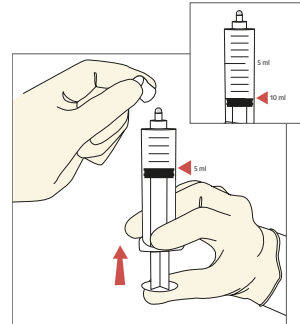


3

- Replace the tip cap by twisting from the end of the syringe.

Note: Be careful not to contaminate the syringe tip.

- Attach the blunt cannula or needle, if required.
- Hold syringe with the tip upwards.
- Tap side for bubbles to rise. Apply and maintain pressure to the plunger until the plunger reaches the labelled flush volume on the syringe (e.g. 10 ml, 5 ml, 3 ml etc.) and air is expelled.

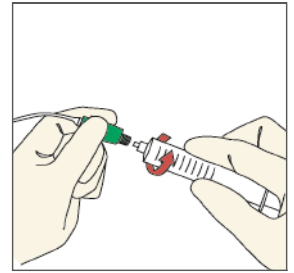

4

- Attach the syringe to the injection site and flush the site with an appropriate volume of solution.

Note: Use the amount of solution your protocol indicates for flushing.

Please note:

This applies to saline syringes only. The device is not intended for the sterile field. It is single-use and should only be used for IV flushing only—aseptically.

Follow Medline's instructions for use. Along with your facility's protocol for administering an IV flush, use in accordance with warnings and precautions that are appropriate to the medication being administered.

Do not use this solution if there is doubt about compatibility. Discard the syringe and any unused parts.

