


How to Use: ReadyFlush® Wipes



Dispersible personal cleansing cloths

Cleanse after every episode of incontinence or use for daily, personal clean-up of all body areas as required.

- 1 Keep ReadyFlush wipes at the bedside or near the toilet.
- 2 Open the package as indicated, use your finger to find the edge of the wipe and then gently pull on it to remove it from the package.
- 3 Wipe all affected areas. When cleaning the perineal region, wipe from front to back. Use additional wipes if needed.
- 4 Dispose of soiled ReadyFlush wipes in the toilet or macerator. Do not ball up wipes or attempt to flush a mass quantity of wipes.
- 5 Close the package after use to maintain the wipes' moistness.

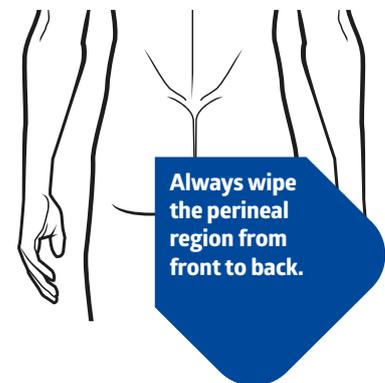

For more information about ReadyFlush, contact your Medline account manager or visit: www.medline.eu/uk.