



HOW TO USE PROLANCE SAFETY LANCETS



Twist off the protective cap and pull it straight out.









Place Prolance on the desired puncture site and press the button to activate it.



Gently apply intermittent pressure near the puncture site to obtain the required blood volume.


Product Listing

Item Number	NPC	Description	Needle	Penetration Depth	Colour
7604	FTM439	Micro flow	Needle 28G	1.6 mm	
7599	FTM440	Low flow	Needle 25G	1.4 mm	
7594	FTM442	Normal	Needle 21G	1.8 mm	
7601	FTM441	High flow	Needle 18G	1.8 mm	
7609	FTM443	Paediatric	Blade	1.2 mm	
7620	FTM448	Max flow	Blade	1.6 mm	

For more information about this product, please contact your account manager or visit us at www.medline.eu/uk

*These products are class IIa medical devices intended to be used by healthcare professionals. Before use, consult instructions and precautions on the corresponding labelling.

 HTL-Strefa S.A.
ul. Adamówek 7
95-035 Ozorków, Poland

 0344 (DEKRA)