



**QUICK AND SIMPLE
TO USE.**

Prolance[®]
Safety lancets for capillary blood sampling.



Item #	Description	Needle	Penetration Depth	Color	Packaging
7604	Micro Flow	Needle 28G	1,6 mm	blue	200/cs
7599	Low Flow	Needle 25G	1,4 mm	dark blue	200/cs
7594	Normal Flow	Needle 21G	1,8 mm	green	200/cs
7601	High Flow	Needle 18G	1,8 mm	yellow	200/cs
7620	Max Flow	Blade	1,6 mm	purple	200/cs
7609	Pediatric	Blade	1,2 mm	pink	200/cs

ADVANTAGES:

- All needles are gamma-sterilised and protected with a sterility cap.
- Fully enclosed needle housing ensures that needle is hidden before and after use to prevent sharps injuries.
- Self-destructing mechanism prevents reuse of the device.
- Silicone-coated, ultra-sharp needles, ideally positioned during skin penetration.
- Various, color-coded product versions to meet specific blood sample applications and address the variety of patient skin types.
- Wide range of designs provides simple, comfortable and secure puncture.
- Easy to handle, intuitive activation in only two steps with no pre-loading, for both push button and contact activations.
- Wide and long safety cap for easy removal – simply twist and pull to remove the needle cover
- Precise design for convenient finger positioning and ideal blood sample collection.

HOW TO USE PROLANCE®?



Twist off the protective cap, and then pull it straight out.



Place Prolance® on the desired puncture site and press the button to activate the device.



Gently apply intermittent pressure near the puncture site to obtain the required blood volume.

For more information on this product, please contact your account manager.



ALWAYS
ON.

Medline Industries Ltd
3rd Floor
Quayside Wilderspool Business Park
Greenalls Avenue
Warrington WA4 6HL
United Kingdom
Tel.: +44 844 334 5237
Fax: +44 844 334 5238

www.medline.eu/uk
uk-customerservice@medline.com