



**QUICK AND SIMPLE
TO USE.**

Prolance[®]
Safety lancets for capillary blood sampling.



Item #	Description	Needle	Penetration Depth	Color	Packaging
7604	Micro Flow	Needle 28G	1,6 mm	blue	200/cs
7599	Low Flow	Needle 25G	1,4 mm	dark blue	200/cs
7594	Normal	Needle 21G	1,8 mm	green	200/cs
7601	High Flow	Needle 18G	1,8 mm	yellow	200/cs
7620	Max Flow	Blade	1,6 mm	purple	200/cs
7609	Pediatric	Blade	1,2 mm	pink	200/cs

HOW TO USE PROLANCE®?



Twist off the protective cap, and then pull it straight out.



Place Prolance® on the desired puncture site and press the button to activate the device.



Gently apply intermittent pressure near the puncture site to obtain the required blood volume.

ADVANTAGES:

- All needles are gamma-sterilised and protected with a sterility cap.
- Fully enclosed needle housing ensures that needle is hidden before and after use to prevent sharps injuries.
- Self-destructing mechanism prevents reuse of the device.
- Silicone-coated, ultra-sharp needles, ideally positioned during skin penetration.
- Various, color-coded product versions to meet specific blood sample applications and address the variety of patient skin types.
- Wide range of designs provides simple, comfortable and secure puncture.
- Easy to handle, intuitive activation in only two steps with no pre-loading, for both push button and contact activations.
- Wide and long safety cap for easy removal – simply twist and pull to remove the needle cover
- Precise design for convenient finger positioning and ideal blood sample collection.

For more information on this product, please contact your Sales Representative or visit us at www.medline.com/uk



ALWAYS
ON

Medline Industries Ltd

No 5 Booths Park
Chelford Road
Knutsford, Cheshire WA16 8GS
England
Tel: +44 844 334 5237
Fax: +44 844 334 5238
www.medline.com/uk
uk-customerservice@medline.com