

How to use the Medline Skin Stapler



Medline's Skin Staplers are designed to be an intuitive, reliable and versatile wound closure solution. They:

- Implement a one-shot firing system
- Contain 35 medical-grade stainless steel staples
- Integrate a transparent window for precise positioning and viewing of the remaining staples
- Provide a rectangular shape to support wound closure stability
- Are available in 2 staple widths.



F-35WB



F-35RB



1

Pull the edges of the wound together until they are everted. Position the Skin Stapler on the midline of the wound. Hold it vertically, with the discharger flat against the surface of the skin.



2

Slowly press the handle and squeeze the trigger fully until an audible 'click' is heard. This indicates the staple's full closure. As the handle is squeezed, an anvil automatically bends the staple to the proper configuration.



3

Be sure to release the trigger before lifting the skin stapler off of the skin.



4

Repeat steps 1 through 3 until the entire wound has been closed.

Item number	Description		Packaging	Pack component number
F-35WB	Stapler with 35 wide staples Staple dimensions after closure: 7.2 x 4.9 mm		5/box, 100/case	9080-EU
F-35RB	Stapler with 35 regular staples Staple dimensions after closure: 5.9 x 3.9 mm		5/box, 100/case	9081-EU

For more information about this product or to receive a product demonstration, please contact your Medline account manager or visit our website: www.medline.eu/uk

These products are class IIa medical devices intended to be used by healthcare professionals. Before use, consult instructions and precautions on the corresponding labelling. All rights reserved. We reserve the right to correct errors that may have occurred.

EC REP Shanghai International Holding Corp. GmbH
Eiffelstraße 90
20537 Hamburg, Germany

© 2022 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP. Three Lakes Drive, Northfield, IL 60093, USA. ML1346_EN 12/2022