



SUREPREP® SPRAY

Liquid Skin Protectant

DIRECTIONS

- 1) Cleanse and dry the skin before use.**
- 2) Hold spray nozzle 4-6 inches (100 to 150mm) from skin.**
- 3) Apply a uniform coating over the entire area of concern.**
 - a) If an area is missed, reapply to that area after barrier spray has dried, approximately 1 minute.
 - b) If applied to an area where skin to skin contact occurs, make sure that the barrier spray dries thoroughly before contact occurs.
 - c) When used as protection from adhesive tapes, dressings or adhesive devices allow barrier spray to thoroughly dry before covering with tape, dressing or adhesive device. Reapplication is necessary each time dressing or adhesive products are changed as the film is removed from the skin by the adhesive.
- 4) Barrier spray can be removed by using most medical adhesive removers or soap and warm water.** Replace cap over spray nozzle when not in use.
- 5) Reapplication is recommended every 24 to 72 hours depending on frequency of cleansing.** In extreme cases (such as constant diarrhoea) with very frequent cleansing more frequent applications may be required (every 12-24 hours).

Item code	Description	UOM
MSC1528	Sureprep spray, 28ml	12/cs



For more information on this product, please contact Vicky Raynor vicky.raynor@medline.com or visit us at www.medline.com/uk



SUREPREP® WIPE/WAND

Liquid Skin Protectant

DIRECTIONS

- 1) It is recommended that gloves are worn for application of No-sting Barrier Wipe/Wand.
- 2) Cleanse and dry the skin as much as possible prior to application.
- 3) Open sachet and remove either the tissue wipe or foam tipped wand applicator.
Evenly and uniformly apply a layer of barrier solution over the entire area of concern.
- 4) Reapply the solution to any area that has not been covered, preferably waiting for the earlier application to dry (approximately 1 minute).
- 5) If applying between skin folds, separate the skin during application and ensure that the product is thoroughly dry before allowing skin to resume its normal position to avoid adhering skin surfaces together.
- 6) Product can be used as a protective barrier between adhesive dressings or fixation tape and the skin. During such use allow the Sureprep No-Sting Barrier Film to thoroughly dry before application of the adhesive tape or dressing. Reapply Sureprep No-Sting Barrier Film after a change of the tape or dressing as the film is removed by the adhesive.
- 7) During use as a protective barrier against bodily fluids, reapply as needed, every 2 to 3 days depending on frequency of cleansing. For heavy fluid exposure, for example during constant diarrhoea, more frequent application may be necessary, such as every 12-24 hours.
- 8) The film barrier can be removed by using most medical grade adhesive removers or by washing with soap and warm water.

Item code	Description	UOM
MSC1505	Sureprep Wipes	500/cs
MSC1510	Sureprep Wand, 1ml	500/cs
MSC1513	Sureprep Wand, 3ml	500/cs



For more information on this product, please contact Vicky Raynor vicky.raynor@medline.com or visit us at www.medline.com/uk