



***REDUCE THE RISK OF
CROSS-CONTAMINATION.***

Disposable Tourniquets
Cost Effective Alternative to Reusable Tourniquets

DISPOSABLE TOURNIQUETS

Disposable Tourniquets from Medline are used on patients for venous blood sampling and intravenous cannulation. Numerous studies show that reusable tourniquets represent a potential source of cross-contamination between patients and healthcare workers due to the insufficient disinfection of tourniquets and the lack of infection control procedures. Use of these tourniquets could be a major cause in spreading of severe, healthcare-associated infections including MRSA.

For the above reasons and in order to help healthcare workers prevent the spread of serious infections, our tourniquets:

- promote a [single-use policy](#),
- offer an unbeatable combination of [value, quality and ease of use](#),
- and also feature a comfortable, [latex-free](#) texture that allows the tourniquet to remain secure while in use.

Instruction for use



1. Place tourniquet around arm, cross the tourniquet and stretch.
2. Stretch the tourniquet and press one end; around the arm, bring the other end under the circle.
3. The tourniquet forms into a bow, but please note that both ends of the tourniquet are on the same side which is important for removal. Do NOT tie a knot as removal will be challenging. Tuck the end under the other so the tourniquet remains on the patient in a secure fashion.
4. Release tourniquet – to remove, pull the end of the tourniquet as indicated on the photo.
5. Dispose of tourniquet in the clinical waste bin after use.

Item	Description	Size	Packaging
DYNDE75020A	Disposable Rolled Tourniquets	2.5 x 45.7 cm	100/cs
DYNDE75025	Disposable Rolled Tourniquets	2.5 x 55 cm	25/cs



For more information on this product, please contact your sales representative or visit us at www.medline.eu/uk



ALWAYS
ON.

Medline Industries Ltd
3rd Floor
Quayside Wilderspool Business Park
Greenalls Avenue
Warrington WA4 6HL
United Kingdom
Tel.: +44 844 334 5237
Fax: +44 844 334 5238

www.medline.eu/uk
uk-customerservice@medline.com

We reserve the right to correct any errors that may occur within this brochure.

© 2019 Medline is a registered trademarks of Medline Industries, Inc. Three Lakes Drive, Northfield, IL 60093, USA. Class I non sterile products according to the MDD 93/42/EEC. ML535-EN02/VM 01/2019.

Medline International France SAS,
5 rue Charles Lindbergh,
44110 Châteaubriant, France