



**CENTURION**  
MEDICAL PRODUCTS

# WingGuard®

Catheter Securement Device

**Patient-friendly,  
nurse-friendly,  
dressing-friendly**



## Benefits for caregivers, hospitals and patients

- Extremely easy to apply and remove
- Universal size secures any size or type of PICC catheter
- Low-profile design minimises bumping and jostling to the site
- Pliable construction conforms comfortably to body contours
- Patient-friendly adhesive is gentle, yet stays in place for long wear times
- Available in latex-free material

WingGuard meets  
INS standards for  
catheter stabilisation  
and manufactured  
stabilisation  
devices

## Application instructions

Thoroughly clean and prepare site. Prep well beyond the entire site according to facility protocol to ensure adherence. Allow it to dry completely (for at least 30 seconds). Note: If the prep doesn't dry completely, the integrity and security may be compromised. A skin protectant may be used according to preference or protocol. Allow the area to dry completely.


Slide WingGuard under the catheter hub and centre the hub wings over WingGuard.

Holding the catheter in place, pull the tab out from the centre to create an opening in the slit. Insert the first catheter wing into the slit.

Repeat on the other side.

Remove the liner one side at a time and adhere WingGuard to the skin.

Apply your preferred dressing over the site as usual.

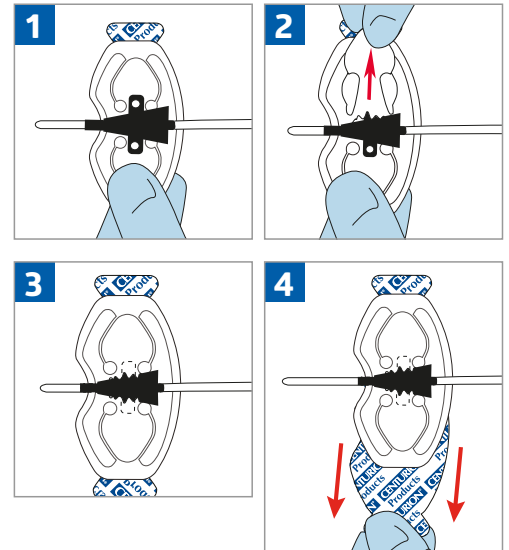

## Removal instructions (not shown)

Remove the dressing.

Optional: Apply a sterile tape strip to catheter tubing to help stabilise the catheter prior to removing WingGuard.

While stabilising the catheter, pull the WingGuard tab out from the centre to create an opening in the slit to release the catheter hub wings from the device.

Peel WingGuard slowly off the skin, away from the exit site.



'Improved adhesive products and securement devices ... are essential to provide increased catheter stability. Such products ... may reduce catheter dislodgement and the necessity of reinsertion with its associated needle stick risk'.

- From the OSHA FactSheet: Securing Medical Catheters



**Medline Industries Ltd**  
3rd Floor  
Quayside Wilderspool Business Park  
Greenalls Avenue  
Warrington WA4 6HL  
United Kingdom  
Tel.: +44 844 334 5237  
[www.medline.eu/uk](http://www.medline.eu/uk)  
[uk-customerservice@medline.com](mailto:uk-customerservice@medline.com)


**Medline Ireland Ltd**  
1 Grant's Row  
Mount Street Lower  
Dublin 2  
Ireland  
Tel.: +353 1 691 73 06  
[www.medline.eu/ie](http://www.medline.eu/ie)  
[ie-customerservice@medline.com](mailto:ie-customerservice@medline.com)

FOLLOW US  

WingGuard securement devices are class I medical devices intended to be used by healthcare professionals. Before use, consult instructions and precautions on the corresponding labelling.



Medline International France SAS  
5 Rue Charles Lindbergh  
44110 Châteaubriant, France

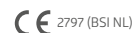

All rights reserved. We reserve the right to correct errors that may have occurred.

© 2022 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP. ML948\_EN 04/2022

WingGuard® is a registered trademark of Centurion Medical Products Corporation. © 2022 Centurion Medical Products Corporation LIT088V3E