

How to apply: Surgical Masks

Recommended to change mask every 3 hours



1 Wash/clean hands



2 Ensure Medline stamp faces outwards and the conformable band is uppermost



3 Conform band to the bridge of your nose



4 Tie upper strap at crown of head



5 Tie lower strap at base of neck



6 Ensure pleats are fully extended to cover nose and mouth



7 If there is risk of splashing, use appropriate eye protection



8 After use, remove mask by tearing straps. Do not let mask rest on your neck



9 Dispose of the mask correctly



10 Wash your hands after disposing of the mask

To learn more about surgical masks, contact your Medline account manager or visit www.medline.eu/uk

