



Fitting Visual-Cueing Slipper Socks Correctly

#thinkyellow #thinkfalls



The soft cuff of the slipper sock holds it in position, preventing the sock from slipping down. The cuff must sit just above the ankle bone, and the tread must extend past the heel.

The fit must be close enough to hold the slipper sock in position, in order to minimise unnecessary movement that could cause friction. However, the sock must not compress the foot, as this may impede circulation.

Reassess sizing at regular intervals, as the patient may require different sizes during his/her stay. Please see the table below for additional sizes of Falls Prevention Slipper Socks.

Additional sizes of Slipper Socks for use with Yellow Falls Prevention Kits

| SKU | NPC code | Size | UK size | Colour | CE |
|--------------|----------|----------|---------|--------|-----|
| MDTES4B03FPY | CVW049 | Medium | 1 – 3 | Yellow | Yes |
| MDTES4B04FPY | CVW050 | Large | 3 – 5 | Yellow | Yes |
| MDTES4B05FPY | CVW051 | X-Large | 5 – 9 | Yellow | Yes |
| MDTES4B06FPY | CVW052 | XX-Large | 9 – 11 | Yellow | Yes |

For additional information on falls prevention and products, please contact your account manager or visit: www.medline.eu/uk