Transport Tray

Instruction for use

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Medline Industries, LP,
Three Lakes Drive, Northfield, IL 60093 USA

EC REP

Medline International France SAS,
5 rue Charles Lindbergh,
44110 Châteaubriant, France

CH REP

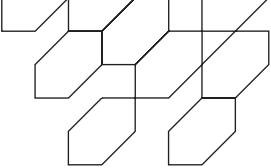
CH Rep and importer:
Medline International Switzerland SARL,
c/o MN & Associés SA, 1 Place de Longemalle,
1204 Genève, Switzerland

UK CA **UK representative and importer in UK:**
Medline Industries Ltd, WA4 6HL,UK

Distributed in Australia by:
Medline International Two Australia Pty. Ltd., Sydney, Australia

REF GEMTRAY3CE

REF GEMTRAY4CE

en

How to use the Medline Gemini sterilization transport tray:

- Identify set on which the Medline Transport Tray will be used. The inner measurements of the trays are either 30cm wide x 56cm long to accommodate full-size instrument baskets or 43cm wide x 70cm long for larger baskets.
- Using your standard procedure, wrap instrument set.
- Place the wrapped instrument set onto the Transport Tray. The set should be transported on the tray until it is ready to be unwrapped in the operating room.
- Bring the Transport Tray containing the wrapped set to the sterilizer following standard hospital procedure. Use the Transport Tray to lift, carry and support the set. It may be used in steam autoclave sterilizers with the recommended parameters of up to 132° C (270° F), an exposure time of 4 minutes and a dry time of 20 minutes.
- After the sterilizer has cooled down, transfer the Transport Tray containing the wrapped set to a sterile storage area. The tray and its contents should remain in this area until it is needed for use in the operating room.
- Set the Transport Tray containing the wrapped set on a case cart or move it according to your standard procedure when needed for use in the operating room.
- When inside the operating room, the wrapped set should be removed from the Transport Tray. After the set has been removed, it can be unwrapped for use.
- Bring the Transport Tray back to the sterile processing area for holding.
- Transport Trays can be reprocessed using the same procedure as the instrument sets. Inspect the tray after use, and discard if the trays are damaged or cracked. Dispose of the product in accordance with hospital, administrative, and local government policy.
- Clean and disinfect the Transport Tray according to general hospital protocols and standards of practice. It may be reused until damaged, deformed or cracked.

Warnings:

- If a serious incident has occurred in relation to the device, it should be reported to Medline and the competent authority of the Member State in which the user and/or patient is established.
- Disposal Instructions: Precautions should be taken when discarding this device and disposal of the device shall be made in accordance with applicable hospital or national regulations for biologically hazardous waste.
- Inspect the Transport Tray before use and discard if the trays are damaged or cracked.

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